

Tristel Duo™ ULT

**Point of Care Ultrasound (POCUS)
has been increasingly adopted across
different disciplines.**

POCUS is increasingly being performed for both invasive and non-invasive procedures. Ultrasound probes are used to assist in the placement of central venous catheters (CVC), to aid the insertion of peripherally inserted central catheters (PICC), anesthetic blocks and premature atrial contraction (PAC), as well as for musculoskeletal procedures (MKS).




TRISTEL DUO ULT IS EASY TO USE. SIMPLY APPLY, WIPE AND WAIT!



1
DISPENSE TWO ALIQUOTS OF TRISTEL
DUO ONTO A DRY WIPE
(DUO WIPE IS RECOMMENDED).



2
USE THE WIPE TO SPREAD THE FOAM
OVER THE SURFACE OF THE ULTRASOUND
PROBE AND ASSOCIATED PARTS.
ENSURE ALL AREAS ARE COVERED.

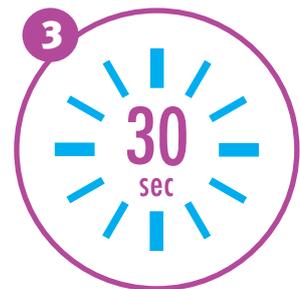

3
LEAVE THE SURFACE TO DRY TO
ENSURE A MINIMUM CONTACT TIME
OF 30 SECONDS.

Tristel Duo ULT is a chlorine dioxide-based foam, suitable for the disinfection of skin surface probes used during POCUS procedures. Tristel Duo ULT is easy to implement in a wide variety of areas where POCUS is performed due to its size and simplicity.

These include:

- Emergency services
- Intensive care units
- General hospital
- Oncology
- Radiology
- Physiotherapy
- portable ultrasound
- Dialysis



EFFECTIVE

Tristel Duo ULT is sporicidal, mycobactericidal, virucidal, fungicidal and bactericidal with a contact time of 30 seconds. The product has been extensively tested according to European Standard suspension tests and a standardised methodology that involves the inoculation of surfaces with the test organism. Tristel's chlorine dioxide is effective against microorganisms of concern such as:

- *Bacillus subtilis*
- *Mycobacterium terrae*
- Human Papillomavirus (HPV)
- Hepatitis B Virus (HBV)
- Human Immunodeficiency Virus (HIV)
- Herpes Simplex Virus Type 1
- *Aspergillus brasiliensis*
- *Candida albicans*
- *Escherichia coli*
- *Neisseria gonorrhoeae*
- *Staphylococcus aureus*
- *Gardnerella vaginalis*
- *Streptococcus agalactiae*
- Carbapenem-resistant Enterobacteriaceae (CRE) *Klebsiella pneumoniae*
- Vancomycin-resistant Enterococci (VRE) *Enterococcus faecium*
- Methicillin-resistant *Staphylococcus aureus* (MRSA)



COMPATIBLE

Tristel Duo ULT has been tested and confirmed to be compatible by major manufacturers, including:

- Anpinion Medical
- BD (Bard Access)
- BK Medical
- Butterfly Network
- Canon Medical Systems
- Carestream
- Esaote
- Exact Imaging
- FUJIFILM Healthcare
- FUJIFILM SonoSite
- GE Healthcare
- Healcerion
- MCube
- Mindray
- MobileODT
- Philips
- Quantel Medical
- Samsung Healthcare
- Siemens Healthineers
- Sonoscape
- Supersonic Imagine
- Verathon



ECONOMICAL

Tristel Duo ULT does not require investment in expensive equipment or service contracts. Two aliquots of foam and one Duo Wipe is all it takes to achieve disinfection to the highest level.



MOBILE

Tristel Duo ULT can be used wherever required, without having to worry about the limitations that automated systems bring. Tristel Duo ULT is also suitable for use in remote locations where electricity and water are not readily available.

Duo Wipes are designed specifically for the application of Tristel Duo ULT.

Duo Wipes are made of 100% non-woven, low-linting polypropylene (18gsm) and pre-cut to prevent shedding when dispensed from the pouch.

The use of Duo Wipes allows the effective release of the disinfectant onto the surface.



TRISTEL DUO ULT

- 125ml Tristel Base Solution (citric acid) + 125ml Tristel Activator Solution (sodium chlorite)

When mixed upon pressing the foam pump, Tristel's proprietary chlorine dioxide chemistry is generated.

- 310 aliquots per bottle/2 bottles per box

DUO WIPES

- 160 wipes per pouch / 320 wipes per pouch
- 2 pouches per carton / 6 pouches per carton

DUO DOK

- Designed to keep Tristel Duo ULT to hand
- 5 per pack

Tristel™

Medical Device
Decontamination

UK Manufacturer: Tristel Solutions Limited, Unit 1b, Lynx Business Park, Fordham Road, Snailwell, Cambridgeshire, CB8 7NY, UK
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com
For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Malaysia: Tristel Malaysia Sdn. Bhd. Registration No. 202001003956 (1360276-D)
Tristel Malaysia Sdn. Bhd. A-25-3A Pinnacle PJ, Lorong Utara C, 46200 Petaling Jaya, Selangor Darul Ehsan, Malaysia - T +603 79317791 - F +603 79319971
E mail-my@tristel.com

Tristel™
WE HAVE CHEMISTRY

Copyright © Tristel Solutions
MR-BIO-2118-2-22/JAN/2024