

LOOKING FOR A COMPLETE, PATIENT-FOCUSED DECONTAMINATION PROCESS IN CARDIOLOGY?



Optimise your decontamination process with the Tristel Trio Wipes System

A comprehensive high-level disinfection process for
Transoesophageal Echocardiography (TEE/TOE) probes.

The Tristel Trio Wipes System enables all parts of a
TEE probe to be high-level disinfected at the point of care.
Including non-immersible parts of the probe such as the
handle, plug and cable.

Tristel Trio[™]
Wipes System

WHY CHOOSE THE TRISTEL TRIO WIPES SYSTEM?



COMPATIBLE

Tristel Trio is tested and approved by major manufacturers.



EFFECTIVE

Tristel Trio is sporicidal, mycobactericidal, virucidal, fungicidal and bactericidal.



QUICK

Tristel Trio provides a full decontamination process, including traceability, in a matter of minutes.



COMPLETE

The Tristel Trio Wipes System is able to high-level disinfect all parts of the TEE probe.



MOBILE

Tristel Trio can be used at the point of care, without the limitations automated systems bring.



ECONOMICAL

No investment required in machines.

THE TRISTEL TRIO WIPES SYSTEM IS EASY TO USE!



CLEAN

Tristel Pre-Clean Wipe

A non-woven wipe impregnated with a triple enzymatic detergent and surfactant. The wipe is used for thorough cleaning of the medical device's surface to remove soil and organic matter.



HIGH-LEVEL DISINFECT

Tristel Sporicidal Wipe

This wipe combined with the Tristel Activator Foam generates chlorine dioxide, a highly effective biocide that delivers high-level disinfection in a uniform contact time of 30 seconds.



RINSE

Tristel Rinse Wipe

A sterile packed, non-woven wipe impregnated with deionised water and low antioxidant content. The wipe is used to remove any remaining chemical residues from the surface of the medical device.



TRACE

The Quality Audit Trail Record Book or 3T (NEW)


Two traceability options for the simple registration of all steps of the process.


POWERED BY TRISTEL CHLORINE DIOXIDE (ClO₂)

The Tristel Sporicidal Wipe utilises Tristel's proprietary chlorine dioxide chemistry (ClO₂), a well-documented and highly effective biocide.

ClO₂ is a powerful oxidizing agent that packs a punch. It destroys pathogens via electron exchange, sequestering electrons from the microorganism's vital structures such as cell walls, membranes, organelles, and genetic materials. This causes a molecular imbalance leading to the microorganism's death.

This mode of action makes ClO₂ highly efficacious, even at low concentrations.

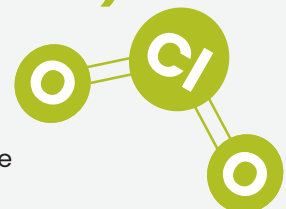

	European norms (EN)	Uniform Contact Time
Bacteria	EN 13727	
	EN 14561	
	EN 16615	
Yeasts/Fungi	EN 13624	
	EN 14562	
	EN 16615	
Viruses	EN 14476	
Mycobacteria	EN 14348	
	EN 14563	
Spores	EN 17126	

Did you know?

EN 17126 is the first standard for the evaluation of sporicidal activity in the medical area. Compliance with this test standard is mandatory to claim sporicidal efficacy. The sporicidal standard, EN 13704, is no longer valid for the medical area. EN 13704 applied to the medical area up until the release of EN 17126 in 2018 as there was no specific standard for this area.

Compliance with EN 17126 requires proven efficacy against *Bacillus cereus* and *Bacillus subtilis* with a minimum reduction of 4 log at a contact time of 60 minutes or less.

For a complete list of micro efficacy data, refer to the Tristel Sporicidal Wipes Microbiological Efficacy Summary.

The Tristel Trio Wipes System is the most widely approved, validated and used high-level disinfection wipes system globally.

NEW

SMART TRACEABILITY

The Tristel Trio Wipes System includes either paper-based or digital traceability. The widely-used Quality Audit Trail Record Book documents each disinfection event with pen and paper. Tristel 3T digitalises this process for a more efficient, accurate and smart approach to traceability.



NOT CONVINCED YET? Try TRISTEL TRIO for your TEE probe

You can count on Tristel Trio:

When your machine breaks down

To eliminate transport
and reduce the risk of damage


To bring reliability to your practice



ORDER DETAILS AND SUPPORTING PRODUCTS

TRISTEL 3T

Available on request.

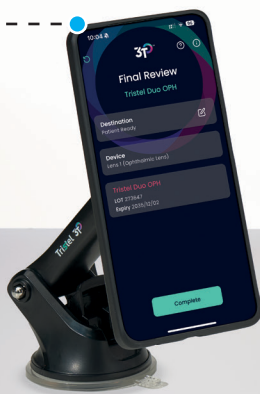

TRISTEL TRIO WIPES SYSTEM

- Available in a box of 50 decontamination procedures, with or without the Quality Audit Trail Record Book.
- Product code: TSL030601.



More information about the Tristel Trio Wipes System, such as safety data sheets, reports, publications on materials and studies, is available on request or online at www.tristel.com/asia-en

Tristel™

Hong Kong: Tristel Asia Limited
T +852 2895 6968 E customerservicehk@tristel.com
Malaysia: Tristel Malaysia Sdn. Bhd.
T +60 37 931 7791 E mail-my@tristel.com
W www.tristel.com/asia-en

India: Tristel India Pvt Ltd
T +91 80417 19452 E mail-india@tristel.com
Singapore: Tristel Pte Ltd
T +65 6518 9918 E mail-sg@tristel.com

Join us on
LinkedIn



Refer to the User Guide for full product instructions.
For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Copyright © Tristel Solutions Limited
Mkt-Bro-3325 - 07/OCT/2025 - VI