



READY TO USE JET LUX? LET'S GET STARTED.

PRE-USE CHECKS

Some important reminders before starting:

- ✓ Always read the label and product information before use.
- ✓ Wear the appropriate personal protective equipment (PPE) when handling disinfectants.
- ✓ Do not use if product is damaged.
- ✓ **JET LUX** is for healthcare professional use only.
- ✓ Do not apply to brass, copper, aluminium, mild steel, or silver surfaces.

HOW TO USE



To assemble **JET LUX**, take a trigger, unscrew the caps on the bottles, and following the **red A** and **blue B** indicators on the bottle label, insert the red tube into the side of the bottle marked with the **red A** and the blue tube into the side of the bottle marked with the **blue B**.




Fasten the levers anti-clockwise on each side of your trigger to secure. Ensure the nozzle is aligned to the top line on the trigger head. **JET LUX** is now ready for use.



Apply at least 2 doses onto a wipe.



Wipe onto the surface.

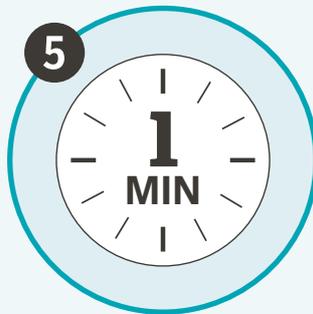

Ensure the surface remains wet for 1 minute. Apply additional solution as needed. Leave to dry.



When you can't recycle, reuse! Read the 'BIG IMPACT' section below to find out more.

BIG IMPACT

Your **JET LUX** trigger should be reused with the second **JET LUX** bottle. To do this, unfasten the levers clockwise to remove the trigger. Before following the assembly instructions above, give the dip tubes a rinse with tap water and dry with a paper towel. After use, empty **JET LUX** bottles and caps can be rinsed out with tap water and sent for recycling.

Refer to the User Guide for full instructions.

STORAGE CONDITIONS

JET LUX has a 2-year shelf life and should be stored in a cool dry place, out of direct sunlight, between 10°C - 35°C. See bottle label for expiry.

Once opened, **JET LUX** has an in-use life of 6 months. You can track this by noting the day you opened **JET LUX** on the bottle label (using permanent ink).

