

READY TO USE JET? LET'S GET STARTED.

PRE-USE CHECKS



HOW TO USE



To assemble **JET**, take a trigger, unscrew the caps on the bottles, and following the red **A** and blue **B** indicators on the bottle label, insert the tubes into the respective bottle necks.




Fasten the levers anti-clockwise on each side of your trigger to secure. Ensure the nozzle is aligned to the top line on the trigger head. **JET** is now ready for use.




Apply at least 2 doses onto a wipe.



Wipe onto the surface.

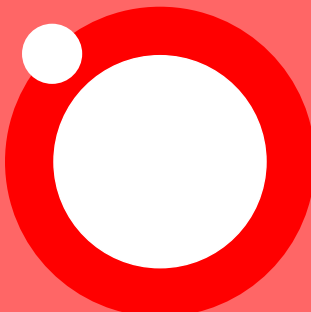

Ensure the surface remains wet for the 1 minute contact time. Leave to air dry.

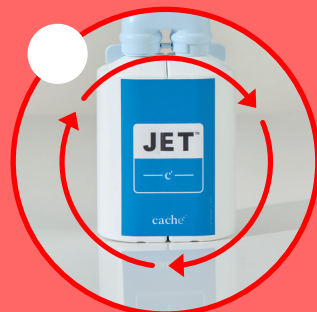

When you can't recycle, reuse! Read the 'BIG IMPACT' section below to find out more.

BIG IMPACT

JET

JET

STORAGE CONDITIONS

JET

JET