

# A TRIPLE-ENZYME FOAM FOR MEDICAL DEVICE CLEANING

Tristel Clean is designed specifically for use on non-lumened invasive and non-invasive medical devices.

Tristel Clean can be used as a stand-alone cleaning product, or prior to disinfection.

Disinfection is only effective when a device has undergone thorough cleaning.

No rinsing is required after using Tristel Clean on its own, or before Tristel disinfectants.



**Tristel**<sup>™</sup>  
**Clean**

**TRISTEL CLEAN IS MOBILE, PRACTICAL & COMPATIBLE**



**TRIPLE ENZYME**  
MEETS CLEANING STANDARDS



**LONG LASTING**  
TWO-YEAR SHELF LIFE



**MOBILE**  
LIGHT-WEIGHT & PRACTICAL



**COMPATIBLE**  
APPROVED BY MAJOR  
MANUFACTURERS



**PRACTICAL**  
NO SOAKING REQUIRED



**FOAM**  
NON-AEROSOL APPLICATION

# TRISTEL CLEAN IS EASY TO USE. SIMPLY APPLY & WIPE.



**DISPENSE TRISTEL CLEAN  
ONTO A DRY WIPE.**

Duo Wipes are recommended.



**SPREAD OVER DEVICE UNTIL SOIL  
AND ORGANIC MATTER HAVE BEEN  
VISIBLY REMOVED.**

In case of heavy soiling repeat the process.



**IF REQUIRED, DISINFECT THE DEVICE IN LINE  
WITH DEVICE APPLICATION REQUIREMENTS.**

Refer to the Tristel disinfectant user guide for full instructions.

## APPROVED BY MANUFACTURERS AND REGIONAL REGULATORY BODIES

Tristel Clean is designed for use on medical devices prior to disinfection with Tristel Duo, Tristel Fuse for Stella or the Tristel Trio Wipes System. It can also be used as a stand-alone cleaning product for procedures such as ultrasound devices.

The Tristel Clean formula has been used for over twenty years in the Tristel Pre-Clean Wipe and is approved for use by major manufacturers such as:

- Advanced Tactile Imaging, Inc.
- Arthrex
- ATMOS MedizinTechnik
- Becton Dickinson (Bard Access, CareFusion)
- BK Medical
- Butterfly Network
- Canon Medical Systems
- Carestream
- Clarius Mobile Health
- Clarus Medical
- DWL Compumedics
- Echosens
- Exact Imaging
- Fisher & Paykel
- Fujifilm (Fujinon)
- Fujifilm (Hitachi)
- FUJIFILM SonoSite
- GE Healthcare
- Haag-Streit
- Karl Storz
- Laborie (Cogentix Medical, MMS)
- Machida Endoscope Co.
- MCube
- Medcom
- Mindray
- Neoplas Tools
- Ocular Instruments Inc
- Optim LLC
- Orvision
- Pentax
- Philips
- Polydiagnost
- Samsung Healthcare
- SapiMed
- Siemens Healthineers
- Southern Scientific / Care Wise
- tic Medizintechnik
- Unisensor
- Verathon
- XION Medical



### TRISTEL CLEAN

310 aliquots per bottle / 6 bottles per box  
310 aliquots per bottle / 2 bottles per box

**Tristel**™

UK Manufacturer: Tristel Solutions Limited, Unit 1b, Lynx Business Park,  
Fordham Road, Snailwell, Cambridgeshire, CB8 7NY, UK  
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

India: Tristel India Private Ltd.  
T +91 88619 23023 - E mail-india@tristel.com - W www.tristel.com

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Contact  
us on  
Whatsapp



Join  
us on  
LinkedIn



Copyright © Tristel Solutions Limited - Mkt-Bro-2966 - 18/DEC/2024 - V2