



Tristel Rinse Assure is a Reverse Osmosis (RO) system specifically designed to be connected to an Endoscope Washer Disinfector (EWD). It combines water filtration and chlorine dioxide ( $\text{ClO}_2$ ) dosing to provide bacteria free water.

Future-proof your endoscopy units against failing rinse water tests.

**Rinse Assure is a 3-in-1 system that:**

- ✓ Filters particles down to  $0.2\mu\text{m}$  through a three-stage filtration process.
- ✓ Treats incoming water through a RO membrane.
- ✓ Doses this water with low levels of Tristel's proprietary chlorine dioxide ( $\text{ClO}_2$ ) chemistry.




## Technical Specifications

### Rinse Assure S2

#### Key Product Features

- ✓ Prevents bacterial proliferation in the incoming water
- ✓ Low dosing of chlorine dioxide
- ✓ Compact design
- ✓ Fully controlled with wireless remote monitoring by a qualified engineer (Subject to service contract)

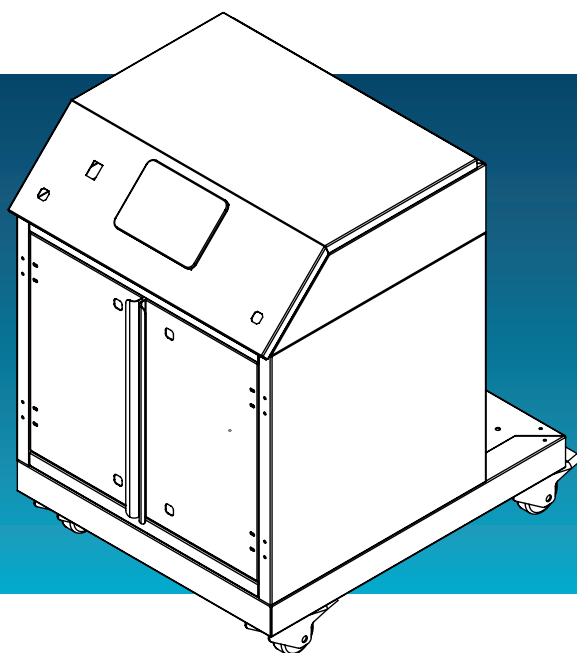

#### Water System Specifications



Parameter	RA-S2
Number of pre-filters	1
Number of final filters	1
Max. Filtration	0.2µm
Disinfectant type	ClO <sub>2</sub>
Chemical container size	2 x 5L
Water delivery temperature	Ambient

#### Feed Water Specifications

Parameter	RA-S2
Min. water inlet temperature (°C)	4
Max. water inlet temperature (°C)	35
Min. water inlet flowrate	18 - 20L/min
Pressure	2-5 Bar
Drainage handling	5 L/min

#### Electrical Requirements


Parameter	RA-S2
Voltage (+/- 10%)	230V AC 50Hz
Socket	13A 3-Pin
HMI Screen size	7 inch

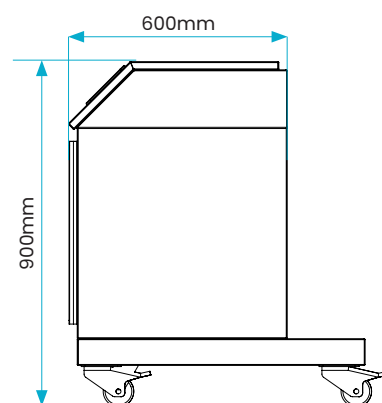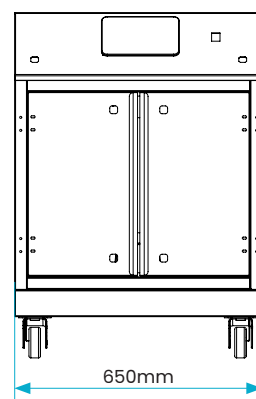

## Rinse Assure S3/3+

### Key Product Features

- ✓ Prevents bacterial proliferation in the incoming water
- ✓ Full reverse osmosis system, including low dosing of chlorine dioxide
- ✓ Total of 55/85 litres dosed storage capacity
- ✓ Fully controlled with wireless remote monitoring by a qualified engineer (Subject to service contract)

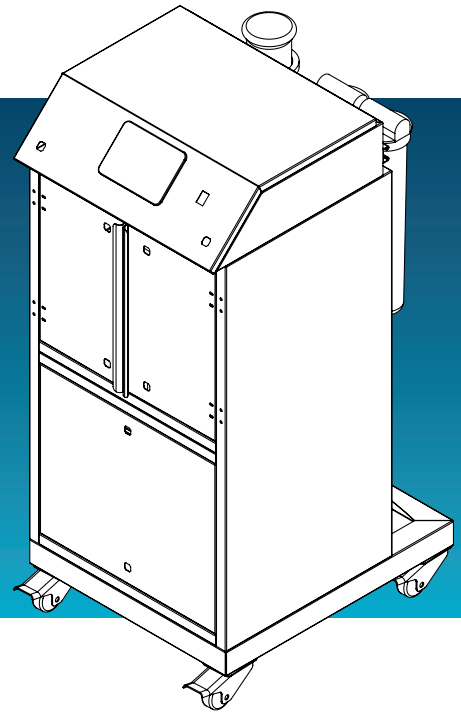


### Water System Specifications

Parameter	RA-S3/3+
Number of pre-filters	2
Number of final filters	1
Max. Filtration	0.2µm
Disinfectant type	ClO <sub>2</sub>
Chemical container size	2 x 5L
No. RO membranes	1x S3 2x S3+
Membrane size (length)	4040 (1016mm)
Membrane diameter	99mm
Tank Capacity	55L (Cat. 5) - S3 85L (Cat5) - S3+
Water production max/hour	210 L/hour - S3 420 L/hour - S3+
Water delivery max. flow rate	25 L/min
Water delivery temperature	Ambient
Salt rejection	99%

### Electrical Requirements

Parameter	RA-S3/3+
Voltage (+/- 10%)	230V AC 50Hz
Socket	13A 3-Pin
HMI Screen size	7 inch

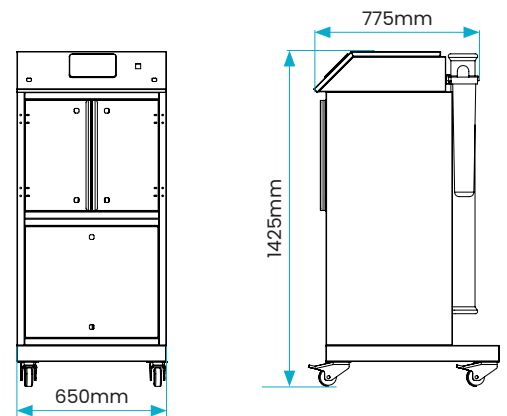

### Feed Water Specifications


Parameter	RA-S3/3+
Min. water inlet temperature (°C)	4
Max. water inlet temperature (°C)	45
Min. water inlet flowrate	10 L/min - S3 15 L/min - S3+
Max. water inlet SDI	25 L/min
Chlorine mg/L	<0.1
Iron/Manganese max. mg/L	<0.1/<0.05
Conductivity	<1000µS/cm
Pressure	2-8 Bar
Drainage handling	10 L/min

## We Have Chemistry

Rinse Assure doses low concentrations of Tristel's proprietary chlorine dioxide ( $\text{ClO}_2$ ) chemistry into the incoming water supply used during an endoscope washer-disinfector (EWD)'s decontamination cycle.\*

Tristel's  $\text{ClO}_2$  has high compatibility with materials used within EWDs at the concentration dosed by Rinse Assure. It does not damage endoscopes and does not affect the chemical composition or efficacy of detergents, including enzymatic, and disinfectants, including peracetic acid, commonly used in EWDs.

\*Dosage levels are adjusted based on the amount of water flowing through the system to ensure bacteria-free water

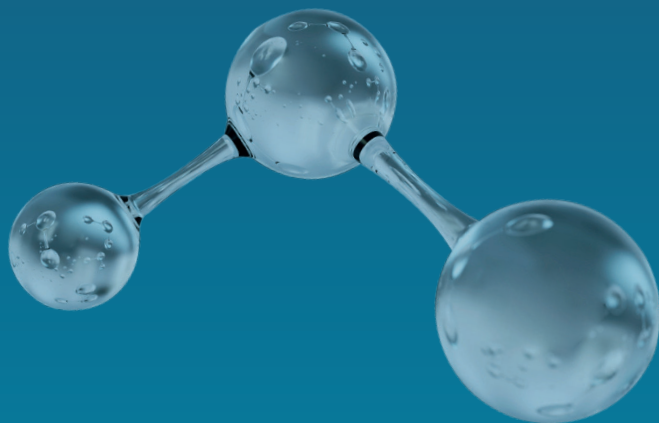

## Proven Effective

Nelson Labs is a leading United States laboratory used by medical device companies, including EWD companies, for disinfectant testing. The effectiveness of Rinse Assure was evaluated by Nelson Labs under Good Laboratory Practice (GLP).

Rinse Assure was challenged with a high bioburden ( $>10^6$  CFU/ml) of *Mycobacterium terrae* and *Pseudomonas aeruginosa*. Organisms were injected directly into the tank of Rinse Assure, representing a worst-case contamination scenario. The Rinse Assure cycle was run, and a  $>6 \log_{10}$  reduction of both *M. terrae* and *P. aeruginosa* was achieved.



## For over 30 years, Tristel has been manufacturing infection prevention products.

With Rinse Assure, we have taken our decades of excellence in chemistry and applied it to rinse water filtration to provide water that:

- ✓ Is bacteria free
- ✓ Contains no pathogens of concern
- ✓ Achieves 0 CFU/mL

# Tristel™

**Hong Kong**  
Tristel Asia Ltd  
T +852 2895 6968  
E customerservicehk@tristel.com

**Singapore**  
Tristel Pte Ltd  
T +65 6518 9918  
E mail-sg@tristel.com

**Malaysia**  
Tristel Malaysia Sdn. Bhd.  
T +60 37 931 7791  
E mail-my@tristel.com

[www.tristel.com/asia-en](http://www.tristel.com/asia-en)



Join  
us on  
LinkedIn



Scan for  
more on  
Rinse Assure