



READY TO USE JET? LET'S GET STARTED.

PRE-USE CHECKS

Some important reminders before starting:

- ✓ Always read the label, safety data sheets (SDS) and product information before use.
- ✓ Wear the appropriate personal protective equipment (PPE).
- ✓ Do not use if product is damaged.
- ✓ **JET** is for professional use only.
- ✓ Do not apply to brass, copper, aluminium, mild steel, or silver surfaces.
- ✓ Carefully clean, rinse and dry the surface before use.

HOW TO USE



To assemble **JET**, take a trigger, unscrew the caps on the bottles, and following the **red A** and **blue B** indicators on the bottle label, insert the tubes into the respective bottle necks.



Fasten the levers anti-clockwise on each side of your trigger to secure. Ensure the nozzle is aligned to the top line on the trigger head. **JET** is now ready for use.




Apply at least 2 doses onto a wipe.



Wipe onto the surface.



Ensure the surface remains wet for the 1 minute contact time. Leave to air dry.

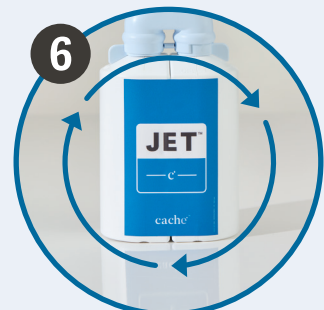

When you can't recycle, reuse! Read the 'BIG IMPACT' section below to find out more.

BIG IMPACT

Your **JET** trigger should be reused with the second **JET** bottle. To do this, unfasten the levers clockwise to remove the trigger. Before following the assembly instructions above, give the dip tubes a rinse with tap water and dry with a paper towel. After use, empty **JET** bottles and caps can be sent for recycling.

STORAGE CONDITIONS

JET can be kept for 2 years before being used, and should be stored in a cool dry place, out of direct sunlight, between 10°C - 35°C.

 Refer to the User Guide for full instructions.