

# JET PRO<sup>c</sup>

## PRE-USE CHECKS

Some important reminders before starting:

- Use biocides safely. Always read the label and product information before use.
- Wear the appropriate personal protective equipment (PPE) when handling disinfectants.
- Do not use if product is damaged.
- JET PRO is for healthcare professional use only.
- Do not apply to brass, copper, aluminium, mild steel, or silver surfaces.
- Ensure the surface is free from heavy soiling before use.

## HOW TO USE



To assemble JET PRO, take a trigger, unscrew the caps on the bottles, and following the red A and blue B indicators on the bottle label, insert the red tube into the side of the bottle marked with the red A and the blue tube into the side of the bottle marked with the blue B.



Fasten the levers anti-clockwise on each side of your trigger to secure. Ensure the nozzle is aligned to the top line on the trigger head. JET PRO is now ready for use.



Apply at least 2 doses onto a dry wipe.



Wipe onto the surface.



Ensure the surface remains wet for 1 minute. Apply additional solution as needed. Leave to dry. JET PRO will continue working to protect your surfaces for up to 8 hours.



When you can't recycle, reuse! Read the 'BIG IMPACT' section below to find out more.

## BIG IMPACT

Your JET PRO trigger should be reused with the second JET PRO bottle. To do this, unfasten the levers clockwise to remove the trigger. Before following the assembly instructions above, give the dip tubes a rinse with tap water and dry with a paper towel. After use, empty JET PRO bottles and caps can be rinsed out with tap water and sent for recycling.



Refer to the User Guide for full instructions.

## STORAGE CONDITIONS

JET PRO has a 2-year shelf life, and should be stored in a cool dry place, out of direct sunlight, between 10°C - 35°C. See bottle label for expiry. Once opened, JET PRO has an in-use life of 6 months. You can track this by noting the day you opened JET PRO on the bottle label (using a permanent ink).

