


HIGH-LEVEL DISINFECTANT FOAM FOR NON-INVASIVE ULTRASOUND DEVICES


NOTES

- Do not use if the bottle is damaged.
- Disinfect hands and wear suitable gloves and Personal Protective Equipment (PPE) when handling disinfectants and medical devices.
- When Tristel DUO ULT is used for the first time, depress the pump two to four times to prime it.
- Clean medical devices thoroughly prior to disinfection. Residual contamination with organic soiling, or ultrasound gel, will decrease the effectiveness of any disinfectant. Follow hospital protocol and medical device manufacturer's instructions.
- Do not use on soft metals or low grades of stainless steel.
- Cotton swabs can be used to apply Tristel DUO ULT into difficult to reach areas such as indentations or crevices.
- For healthcare professional use only.
- This wall chart should be used in conjunction with the instructions for use (IFU), product label, Safety Data Sheet (SDS) and device manufacturer's instructions.

DIRECTIONS FOR USE

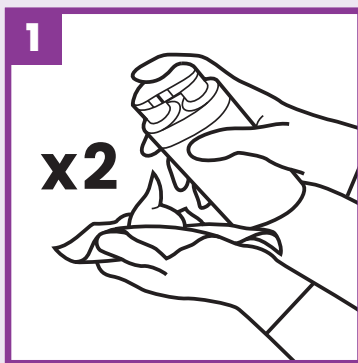


Dispense two aliquots of Tristel DUO ULT onto a dry wipe (DUO WIPES are recommended).

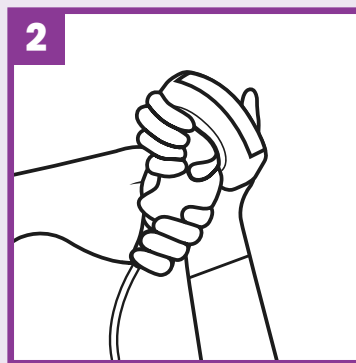

Use the wipe to spread the foam over the surface of the device and ensure all areas are covered. If required, use a cotton swab to spread the foam into indentations.

Note: Discard the used wipe in accordance with local regulations. Do not re-use.

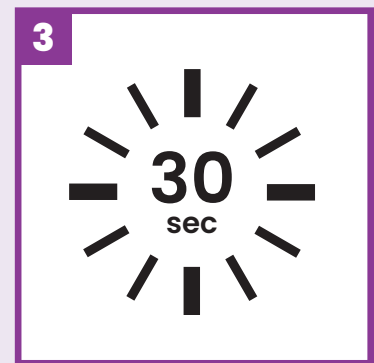

Ensure the device remains wet for a minimum contact time of 30 seconds. Leave to air dry.

Note: Store the medical device in accordance with hospital protocols to prevent damage or recontamination.