

HIGH-LEVEL DISINFECTANT FOAM FOR OPHTHALMIC LENSES


NOTES

- Do not use if the bottle is damaged.
- Disinfect hands and wear suitable gloves and Personal Protective Equipment (PPE) when handling disinfectants and medical devices.
- When Tristel DUO OPH is being used for the first time, depress the pump two to four times to prime it.
- Clean medical devices thoroughly prior to disinfection. Residual contamination with organic soiling will decrease the effectiveness of any disinfectant. Follow hospital protocol and medical device manufacturer's instructions.
- Do not use on soft metals or low grades of stainless steel.
- This wall chart should be used in conjunction with the instructions for use (IFU), product label, Safety Data Sheet (SDS) and device manufacturer's instructions.
- For healthcare professional use only.

DIRECTIONS FOR USE




Lay a wipe (DUO WIPES are recommended) in the palm of your hand and apply two aliquots of Tristel DUO OPH.

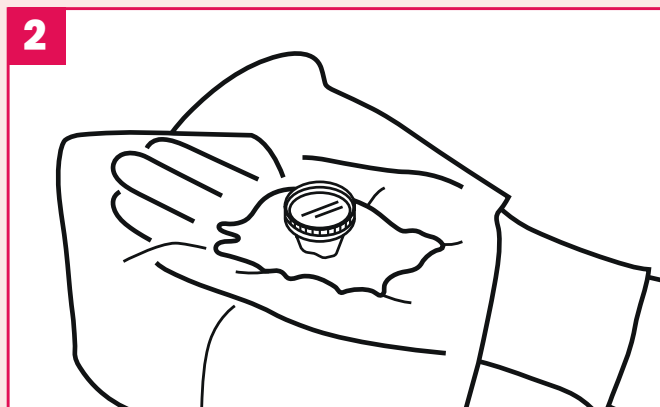


Place the device directly into the foam and use the wipe to ensure all areas of the device are covered.

Note: Discard the used wipe in accordance with local regulations. Do not re-use.

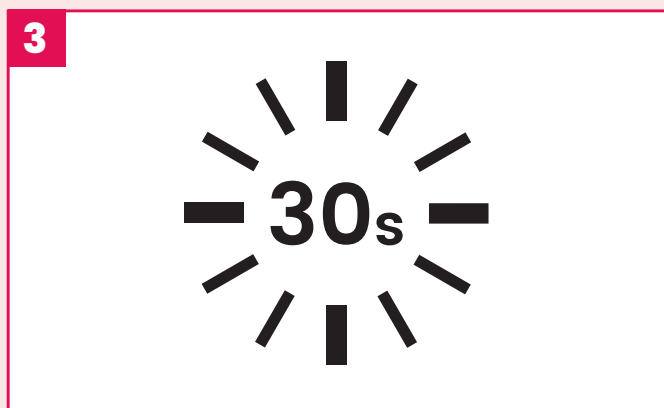

Ensure the device remains wet for a minimum contact time of 30 seconds. Leave to air dry.

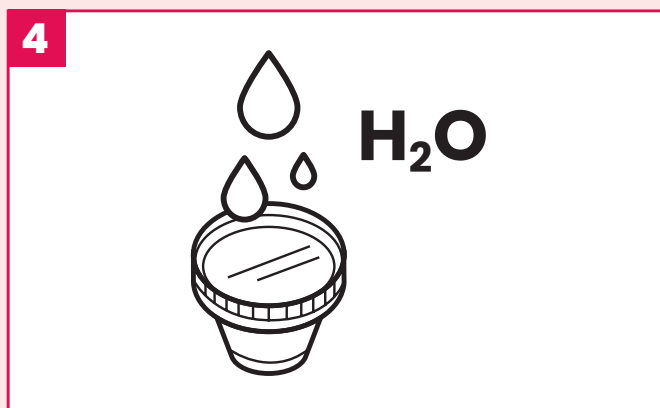

Rinse the contact lens: Rinse the lens with filtered water, sterile water or distilled water that complies with hospital guidelines.

Note: Storage or use: The lens is ready to use or store in accordance with hospital protocol to prevent damage or recontamination.