

HIGH-LEVEL DISINFECTANT FOAM FOR OPHTHALMIC ULTRASOUND PROBES


NOTES

- Do not use if the bottle is damaged.
- Disinfect hands and wear suitable gloves and Personal Protective Equipment (PPE) when handling disinfectants and medical devices.
- When Tristel DUO OPH is being used for the first time, depress the pump two to four times to prime it.
- Clean medical devices thoroughly prior to disinfection. Residual contamination with organic soiling will decrease the effectiveness of any disinfectant. Follow hospital protocol and medical device manufacturer's instructions.
- Do not use on soft metals or low grades of stainless steel.
- This wall chart should be used in conjunction with the instructions for use (IFU), product label, Safety Data Sheet (SDS) and device manufacturer's instructions.
- For healthcare professional use only.

DIRECTIONS FOR USE




Dispense two aliquots of Tristel DUO OPH onto a dry wipe (DUO WIPES are recommended).

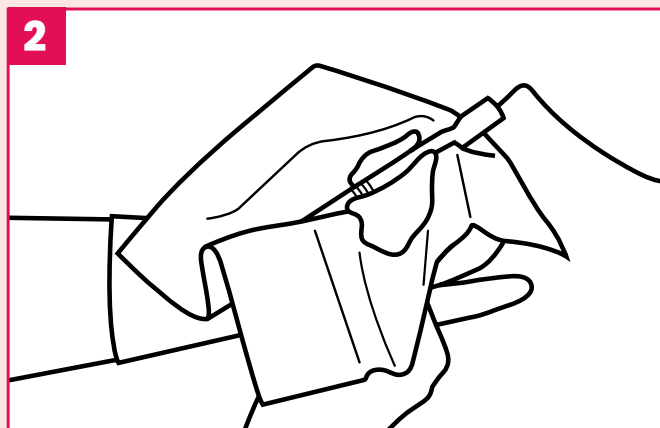


Use the wipe to spread the foam over the surface of the medical device and ensure all areas are covered.

Note: Discard the used wipe in accordance with local regulations. Do not re-use.

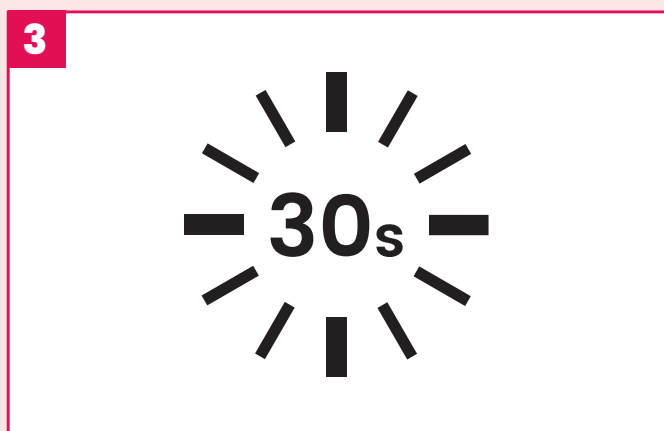

Ensure the device remains wet for a minimum contact time of 30 seconds. Leave to air dry.

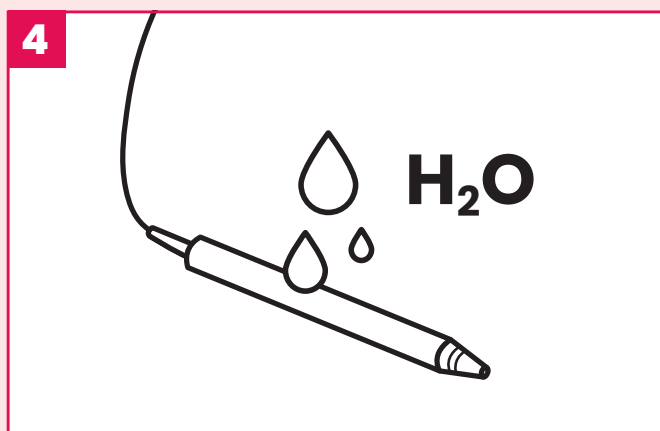

Rinse the ultrasound probe with water of appropriate quality.

Note: Dry and store the medical device in accordance with hospital protocols to prevent damage or recontamination.