

# A TRIPLE-ENZYME FOAM FOR MEDICAL DEVICE CLEANING

Tristel Clean is designed specifically for use on non-lumened invasive and non-invasive medical devices.

Tristel Clean can be used as a stand-alone cleaning product, or prior to disinfection.

Disinfection is only effective when a device has undergone thorough cleaning.

No rinsing is required after using Tristel Clean on its own, or before Tristel disinfectants.



**Tristel**<sup>™</sup>  
**Clean**

## TRISTEL CLEAN IS MOBILE, PRACTICAL & COMPATIBLE



### TRIPLE ENZYME

Meets cleaning standards



### LONG LASTING

Two-year shelf life



### MOBILE

Light-weight & practical



### COMPATIBLE

Approved by major manufacturers



### PRACTICAL

No soaking required



### FOAM

Non-aerosol application

**TRISTEL CLEAN PASSES LOCAL REGULATORY REQUIREMENTS FOR AUSTRALIA & NEW ZEALAND**

# TRISTEL CLEAN IS EASY TO USE. SIMPLY APPLY & WIPE.



**1**  
Dispense Tristel Clean onto a dry wipe.  
Duo Wipes are recommended.



**2**  
Spread over device until soil and organic matter have been visibly removed.  
In case of heavy soiling repeat the process.



**3**  
If required, disinfect the device in line with device application requirements.  
Refer to the Tristel disinfectant user guide for full instructions.

## APPROVED BY MANUFACTURERS AND REGIONAL REGULATORY BODIES

Tristel Clean is designed for use on medical devices prior to disinfection with Tristel Duo, Tristel Fuse for Stella or the Tristel Trio Wipes System. It can also be used as a stand-alone cleaning product for procedures such as non-critical ultrasounds.

The Tristel Clean formula has been used for over twenty years in the Tristel Pre-Clean Wipe and is approved for use by major manufacturers such as:

- Advanced Tactile Imaging, Inc.
- Arthrex
- ATMOS MedizinTechnik
- Becton Dickinson (Bard Access, CareFusion)
- BK Medical
- Butterfly Network
- Canon Medical Systems
- Carestream
- Clarius Mobile Health
- Clarus Medical
- DWL Compumedics
- Echosens
- Exact Imaging
- Fisher & Paykel
- Fujifilm (Fujinon)
- Fujifilm (Hitachi)
- FUJIFILM SonoSite
- GE Healthcare
- Haag-Streit
- Karl Storz
- Laborie (Cogentix Medical, MMS)
- Machida Endoscope Co.
- MCube
- Medcom
- Mindray
- Neoplas Tools
- Ocular Instruments Inc
- Optim LLC
- Orlvision
- Pentax
- Philips
- Polydiagnost
- Samsung Healthcare
- SapiMed
- Siemens Healthineers
- Southern Scientific / Care Wise
- tic Medizintechnik
- Unisensor
- Verathon
- XION Medical

### DUO WIPES

- 200 wipes per tub / 6 tubs per box



### TRISTEL CLEAN

- 310 aliquots per bottle / 6 bottles per box

**Tristel**™

#### Australia:

Tristel Pty Ltd, Unit 44, 328 Reserve Road, Cheltenham, VIC 3192, Australia  
T 1300 680 898 E mail-au@tristel.com W www.tristel.com/au-en

#### New Zealand:

Tristel New Zealand Ltd, 23 Birch Avenue, Judea, Tauranga 3110, New Zealand  
T +64 (0)7 5771560 E mail-nz@tristel.com W www.tristel.com/nz-en/