



READY TO USE JET PRO? LET'S GET STARTED.

PRE-USE CHECKS

Some important reminders before starting:

- ✓ Use disinfectants safely. Always read the label and product information before use.
- ✓ Wear the appropriate personal protective equipment (PPE) when handling disinfectants.
- ✓ Do not use if product is damaged.
- ✓ **JET PRO** is for healthcare professional use only.
- ✓ Do not apply to brass, copper, aluminium, mild steel, or silver surfaces.
- ✓ Ensure the surface is free from heavy soiling before use.

HOW TO USE



To assemble **JET PRO**, take a trigger, unscrew the caps on the bottles, and following the **red A** and **blue B** indicators on the bottle label, insert the red tube into the side of the bottle marked with the **red A** and the blue tube into the side of the bottle marked with the **blue B**.



Fasten the levers anti-clockwise on each side of your trigger to secure. Ensure the nozzle is aligned to the top line on the trigger head. **JET PRO** is now ready for use.



Apply at least 2 doses onto a dry wipe.



Wipe onto the surface.



Ensure the surface remains wet for 2 minutes. Apply additional solution as needed. Leave to dry. **JET PRO** will continue working to protect your surfaces for up to 8 hours.



When you can't recycle, reuse! Read the 'BIG IMPACT' section below to find out more.

BIG IMPACT


Your **JET PRO** trigger should be reused with the second **JET PRO** bottle. To do this, unfasten the levers clockwise to remove the trigger. Before following the assembly instructions above, give the dip tubes a rinse with tap water and dry with a paper towel. After use, empty **JET PRO** bottles and caps can be rinsed out with tap water and sent for recycling.

 Refer to the User Guide for full instructions.

STORAGE CONDITIONS

JET PRO has a 2-year shelf life and should be stored in a cool dry place, out of direct sunlight, between 10°C - 35°C. See bottle label for expiry.

Once opened, **JET PRO** has an in-use life of 6 months. You can track this by noting the day you opened **JET PRO** on the bottle label (using permanent ink).

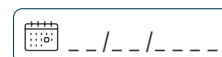

Tristel™

Created by: Tristel Solutions Limited
Unit 1b, Lynx Business Park, Fordham Road,
Snailwell, Cambridgeshire, CB8 7NY, UK
T +44 (0) 1638 721500 E mail@tristel.com
W www.tristel.com

Australian Sponsor:
Tristel Pty Limited, Unit 44,
328 Reserve Road, Cheltenham, VIC 3192, AU
T 1300 680 898 E mail-au@tristel.com
W www.tristel.com/au-en/

New Zealand:
Tristel New Zealand Limited,
23 Birch Avenue, Judea, Tauranga 3110, NZ
T +64 (0)7 5771560 E mail-nz@tristel.com
W www.tristel.com/nz-en/