



THE TRISTEL TRACEABILITY APP

HOW TO UPDATE



TURN ON
THE PHONE



CONNECT
TO WIFI



TAP
'APP CATALOG'
=>'TRISTEL APP'
=>'UPDATE'



WAIT
FOR A COUPLE OF MINUTES
WHILST THE UPDATE IS
DOWNLOADING



COMPLETE
THE UPDATE BY TAPPING
NEXT/OK WHEN ASKED TO
INSTALL



INPUT
YOUR APP INSTALLATION
CODE TO START USING
THE UPDATED APP.*

*If unsure, please contact your organisation's Super User or Tristel to recover your App Installation Code.

AVAILABLE NOW IN:

United Kingdom: Tristel Solutions Limited, Lynx Business Park, Cambis, UK, CB8 7NY
T +44 (0) 1638 721500 - W www.tristel.com

Ireland: Brennan & Company, 61 Birch Avenue, Stillorgan Industrial Park, Stillorgan,
Co. Dublin, A94 XW68 - T 01 2952501 - E enquiries@brennanco.ie - W www.brennanco.ie

Hong Kong: Tristel Asia Ltd, 21st Floor, 168 Electric Road, Fortress Hill, Hong Kong
T +852 2895 6968 - F +852 2869 4388 - E customerservice@tristel.com.hk

Australia: Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192
T 1300 680 898 - F +61 (0)3 9533 6193 - E mail-au@tristel.com

New Zealand: Tristel New Zealand Ltd, 23 Birch Avenue, Judea, Tauranga,
P.O. Box 3110 - T +64 (0)7 5771560 - F +64 (0)7 5771567 - E info@tristel.co.nz

For any enquiries, please contact your
local Tristel representative or email
AppSupport@tristel.com

Developed By:

Tristel