

<b>Mycobactericidal</b>	EN 14563	Carrier	<i>Mycobacterium avium</i>	MGS	Dirty 1	30 sec.
			<i>Mycobacterium terrae</i>	MGS	Dirty 1	

<b>Virucidal</b>	EN 14476	Suspension	Poliovirus Type 1	Dr Brill & Dr Steinmann	Clean 1	30 sec.
			Adenovirus Type 5	Dr Brill & Dr Steinmann	Clean 1	
			Norovirus (using murine norovirus surrogate)	Dr Brill & Dr Steinmann	Clean 1	
			Influenza (H1N1)	Eurofins	Dirty 4	
	ASTM E-1053	Surface	Poliovirus Type 1	Accuratus Laboratories	Dirty 2	
			Adenovirus Type 5	Accuratus Laboratories	Dirty 2	
			Herpes Simplex Virus Type 1	Accuratus Laboratories	Dirty 2	
	DVV and RKI	Suspension	Poliovirus Type 1	Mikrolab GMBH	Clean 2	
				Mikrolab GMBH	Dirty 3	
			Adenovirus Type 5	Mikrolab GMBH	Clean 2	
				Mikrolab GMBH	Dirty 3	
			Human Papillomavirus (using polyoma virus SV40 surrogate)	Mikrolab GMBH	Clean 2	
Mikrolab GMBH				Dirty 3		
Vaccinia virus	Mikrolab GMBH	Clean 2				
	Mikrolab GMBH	Dirty 3				

<b>Fungicidal/Yeasticidal</b>	EN 14562	Carrier	<i>Candida albicans</i>	Biotech Germande	Clean 1	30 sec.
			<i>Aspergillus brasiliensis</i> (formerly <i>niger</i> )	Hygiene Nord GMBH	Clean 1	
	EN 13624	Suspension	<i>Candida albicans</i>	Hygiene Nord GMBH	Clean 1	
			<i>Aspergillus brasiliensis</i> (formerly <i>niger</i> )	MGS	Clean 1	

<b>Bactericidal</b>	EN 14561	Carrier	<i>Staphylococcus aureus</i>	Biotech Germande	Clean 1	30 sec.
			<i>Pseudomonas aeruginosa</i>	Biotech Germande	Clean 1	
			<i>Enterococcus hirae</i>	Biotech Germande	Clean 1	
			Carbapenemase-resistant Enterobacteriaceae (CRE) <i>Klebsiella pneumoniae</i>	MGS	Clean 1	
			Vancomycin-resistant Enterococci (VRE) <i>Enterococcus faecium</i>	MGS	Clean 1	
	EN 13727	Suspension	<i>Staphylococcus aureus</i>	HIRL	Dirty 4	
			<i>Pseudomonas aeruginosa</i>	HIRL	Dirty 4	
			<i>Enterococcus hirae</i>	HIRL	Dirty 4	

**Clean/Dirty Conditions Key**

**Clean 1:** 0.3 g/L bovine albumin  
**Clean 2:** Aqua bidest

**Dirty 1:** 5% Blood Serum  
**Dirty 2:** 5% Fetal Bovine Serum  
**Dirty 3:** 10% Fetal Calf Serum  
**Dirty 4:** 0.3% albumin + 0.3% sheep erythrocytes



**Created by:** Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY  
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

**Australian sponsor:** Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192  
T 1300 680 898 - F +61 (0)3 9533 6193 - E mail-au@tristel.com

**New Zealand sponsor:** Tristel New Zealand Limited, P.O Box 15451, Tauranga, New Zealand  
T +64 (0) 7 5771560 - E info@tristel.co.nz

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Copyright © Tristel Solutions  
Mkt-Mic-584-2  
October 2018