

Tristel Duo™

NCU

FOR NON-CRITICAL
ULTRASOUND PROBES

Intermediate Level Disinfectant

Notes:

- Do not use if the bottle is damaged.
- Disinfect hands and wear appropriate Personal Protective Equipment (PPE) when handling disinfectants and medical devices.
- If Tristel Duo NCU is used for the first time, depress the pump two to four times to prime it.
- Not to be used on soft metals such as brass or copper.
- For professional use.
- This wall chart should be used in conjunction with the instructions for use (IFU), product label, Safety Data Sheet (SDS) and device manufacturer's instructions.

STEP 1




Depress the pump to dispense Tristel Duo onto a dry wipe (Duo Wipe is recommended) or directly onto the surface.

STEP 2




Use the wipe to spread the foam over the surface and ensure all areas are wet.

STEP 3

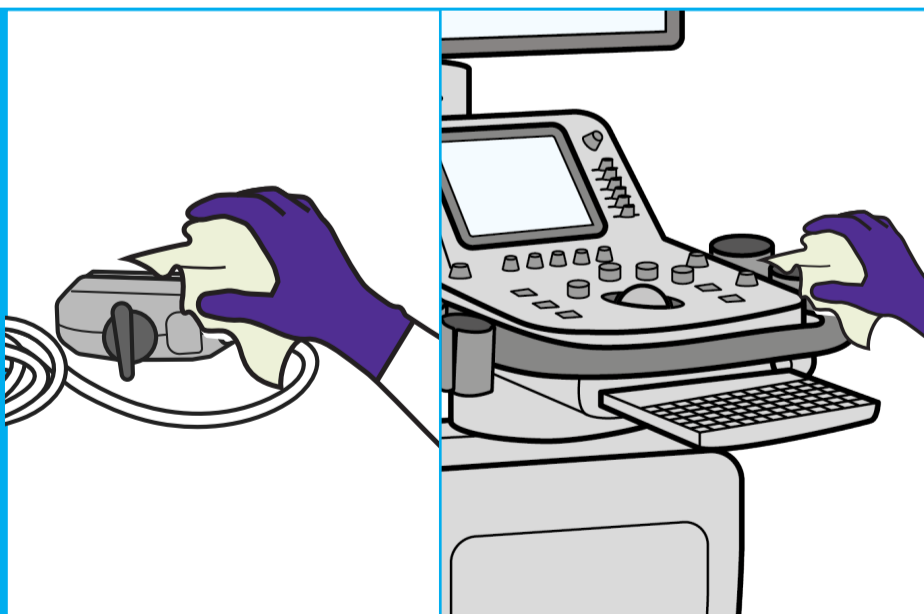

Use Tristel Duo NCU to disinfect surfaces that can come into contact with the probe, such as holders, cables and plugs.

STEP 4

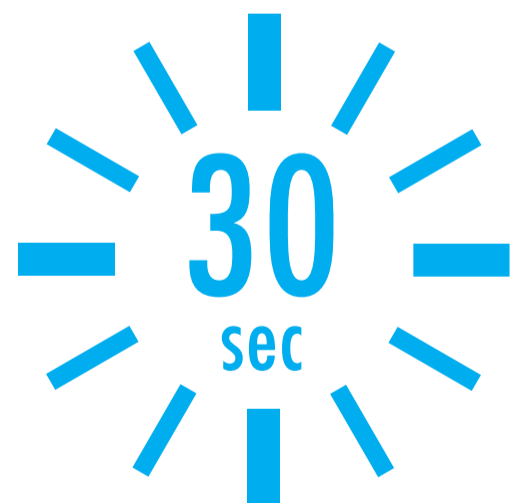

Allow the surface to remain wet for **30 seconds**. Leave to air dry.

Note: Discard the used wipe to clinical waste. Do not macerate. Do not re-use.

Tristel™

Created by: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

Australian sponsor: Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192
T 1300 680 898 - F +61 (0)3 9533 6193 - E mail-au@tristel.com

New Zealand sponsor: Tristel New Zealand Limited, 23 Birch Avenue,
Tauranga 3110, New Zealand - T +64 (0) 7 5771560 - E info@tristel.co.nz

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Copyright © Tristel Solutions
Mkt-Wal-508-3
February 2020