



Reusable Hygienic Tourniquet


STEP 1



APPLICATION

Loop daisygrip around the patient's limb, click magnetic clasp together.

STEP 2

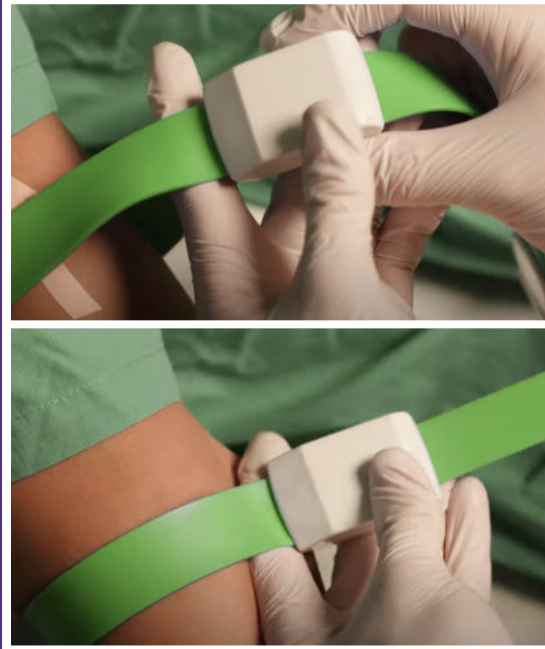

TIGHTENING

Place finger between strap and skin to prevent pinching. Pull strap from close to the clasp at a 45° angle to tighten – do not pull from the distal (tail end) of the strap.

STEP 3



LINE INSERTION/BLOOD COLLECTION

Loosen the daisygrip once the needle is in the vein.


STEP 4



LOOSENING

Place finger under clasp. Use a 'lifting' motion to flip the clasp in the direction of the strap to loosen daisygrip and release stasis.


STEP 5

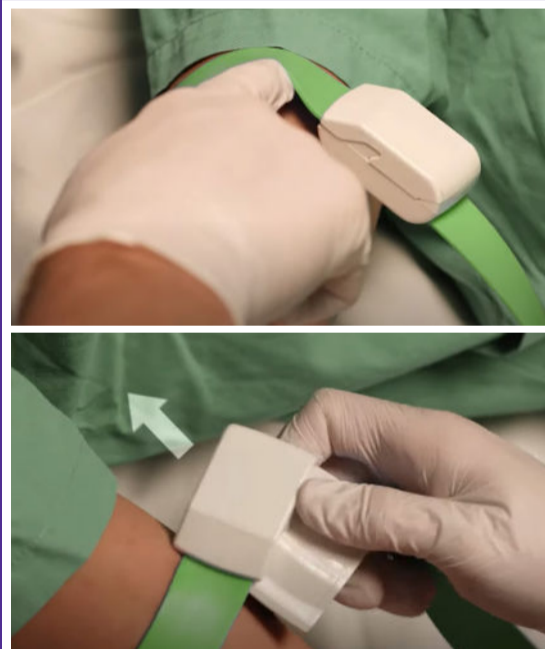

REMOVING

Loosen, then slide the top clasp sideways with your thumb, whilst holding the bottom arm of the clasp steady (with fingers or palm).

STEP 6

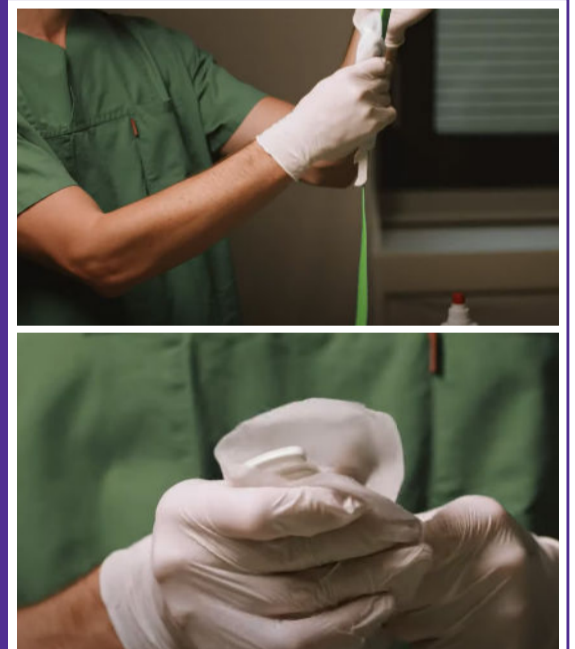

REPROCESSING

Ensure clasp and strap are cleaned and/or disinfected in line with local requirements after every use. Strap should be dry before next use.