
Tristel™

Clean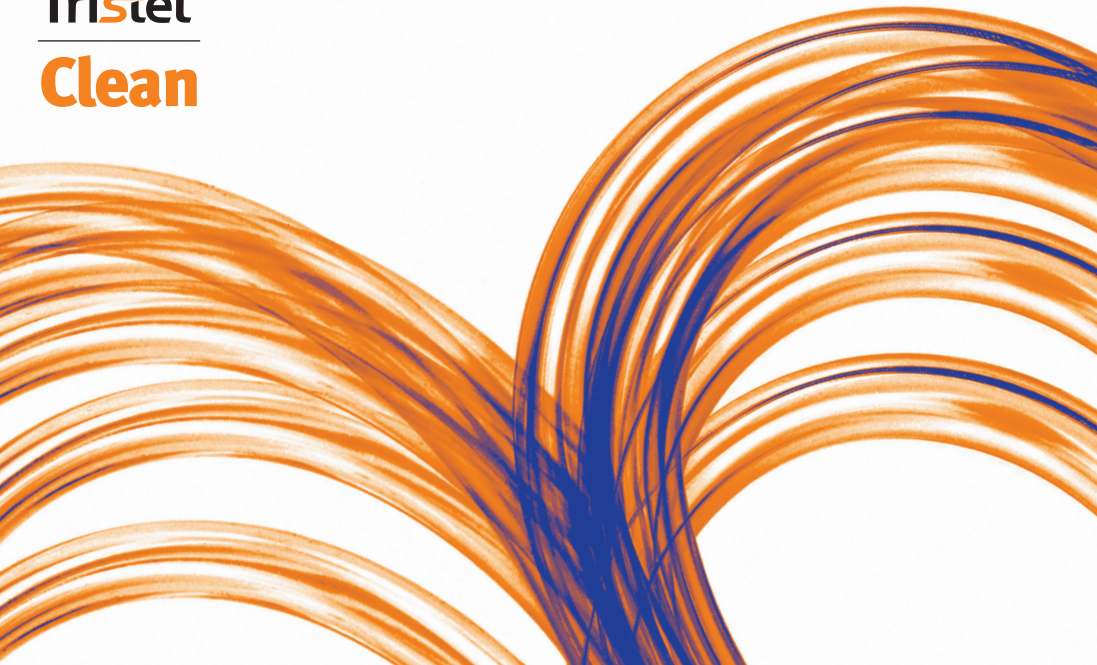

INTENDED USE

Tristel Clean is a triple-enzymatic cleaning foam for use on non-lumened invasive and non-invasive medical devices.

PRECAUTIONARY STATEMENTS

- Do not use if the bottle is damaged.
- Disinfect hands and wear appropriate Personal Protective Equipment (PPE) when handling medical devices.
- For professional use only.
- If Tristel Clean is being used for the first time, depress the pump two to four times to prime it.
- These Instructions for Use (IFU) should be used in conjunction with the product label, Safety Data Sheet (SDS) and device manufacturer's instructions.

Tristel Clean is not classified as hazardous according to GB and EU CLP regulations.

USER INSTRUCTIONS

Step 1

Dispense Tristel Clean onto a dry wipe. (Duo Wipes are recommended.)

Step 2

Use enough foam to cover the entire surface of the device and wipe until visibly clean. Use additional foam and wipes if necessary.

Note: Discard the used wipe and gloves in accordance with local regulations. Do not macerate. Do not reuse.

Step 3

Once visibly clean, the device is ready for disinfection as required by device/hospital protocol.

Note: When subsequent disinfection utilises Tristel Duo, intermediate rinsing is not required. For other disinfection methods, intermediate rinsing may be required to fully remove residues to avoid chemical interactions. Please follow manufacturers' protocol.

If you believe that this device has contributed to a serious incident (i.e. a death, the temporary or permanent serious deterioration of a person's state of health, or a serious public health threat), contact the device manufacturer and your national competent authority immediately.

For Tristel patent information please visit

www.our-patents.info/tristel

Contact Tristel, your local distributor or visit **www.tristel.com** for supporting documents such as Safety Data Sheets.

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IMPORTERS

EUROPEAN UNION (EU) / EUROPEAN ECONOMIC AREA (EEA) / NORTHERN IRELAND / SWITZERLAND / TURKEY

AUSTRIA, BULGARIA, CROATIA, CZECH REPUBLIC, ESTONIA, GERMANY, GREECE, HUNGARY, LATVIA, LITHUANIA, POLAND, SLOVAKIA, SLOVENIA

Tristel GmbH

Karl-Marx-Allee 90A, 10243 Berlin, Germany - **T** +49 (0)30 5484422
E deutschland@tristel.com - **W** www.tristel.de

BELGIUM, FRANCE, LUXEMBOURG, NETHERLANDS

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NORTHERN IRELAND

BRENNAN & CO NI

Unit 6 & 7, Carrowreagh Business Park, Dundonald,
Belfast, Northern Ireland, BT16 1QQ - **T** +44 2890 410 136
E info@brennanconi.com - **W** www.brennanconi.com

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Tristel™

WE HAVE CHEMISTRY.

Tristel Solutions Ltd, Lynx Business Park, Fordham Road, Snailwell, CB8 7NY, UK
T +44 (0)1638 721500 - E mail@tristel.com - W www.tristel.com

Australian sponsor: Tristel Pty Ltd, 44/328 Reserve Road, Cheltenham, VIC 3192, AU T T
T 1300 680 898 - E mail-au@tristel.com - W www.tristel.com

New Zealand: Tristel New Zealand Ltd, 23 Birch Avenue, Tauranga 3110, NZ
T +64 (0)7 5771560 - E info@tristel.co.nz - W www.tristel.com

Tristel NV, Smalandlaan 14B,
2660 Antwerp, Belgium

Tristel AG, Sandgrube 29,
9050 Appenzell, Switzerland

Where necessary, external translations have been sourced from ISO 17100:2015 approved supplier.

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TRISTEL AG
VINGMED AB
YEFE DIS TICARET LTD STI.