
Tristel™ Clean
for Stella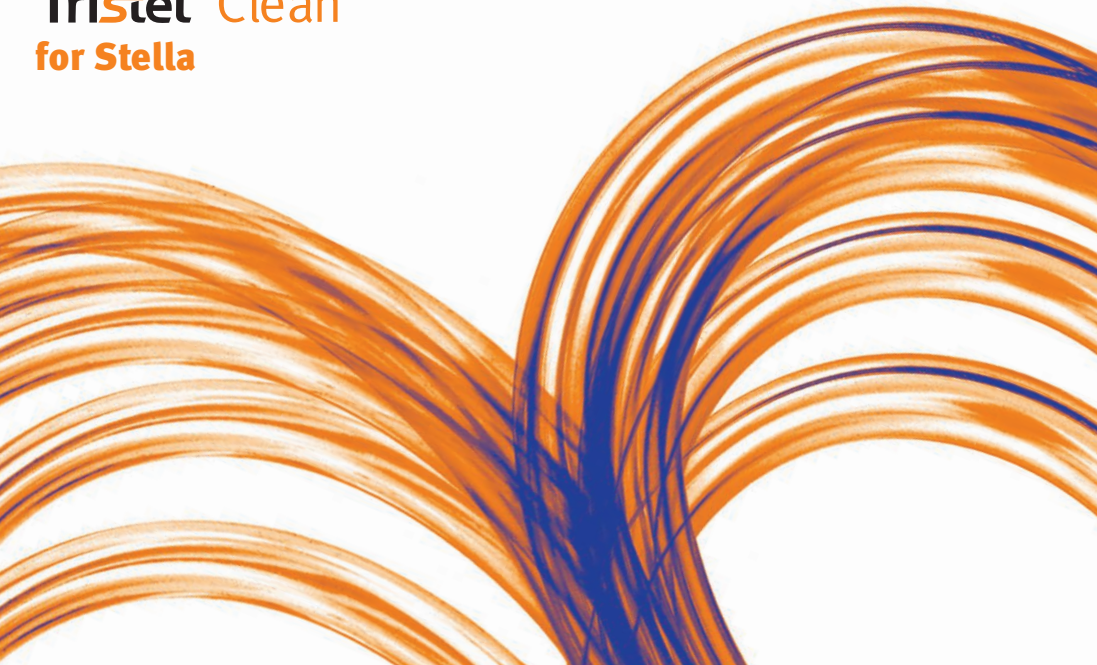

USER INSTRUCTIONS

For professional use only.

Do not use if bottle is damaged.

- These Instructions for Use (IFU) should be used in conjunction with the product label, Safety Data Sheet (SDS) and device manufacturer's instructions.
- Disinfect hands and wear appropriate Personal Protective Equipment (PPE) when handling medical devices.

Clean the medical device directly after use to remove gross contamination.

Preparation of working solution

- Prepare 25ml of Tristel Clean concentrate by squeezing the bottle until the marking on the cap is reached.
- Pour the 25ml concentrate into a suitable vessel (Stella Jug recommended) containing 5L of ambient water.

Note: Do not use hot water.

For manual (pre-) cleaning

Pour the Tristel Clean working solution into a sink.

- Place the medical device into the sink and clean it according to hospital procedures.
- If the medical device has a channel, brush it thoroughly in the cleaning solution. The manufacturer's instructions for the respective device apply.
- Observe a contact time of 5 minutes.
- Remove the medical device from the solution and remove excess liquid.
- Discard Tristel Clean working solution after each use.

For automated cleaning in Stella C

- When Stella IQ displays it, pour Tristel Clean working solution into the device compartment of the Base where the medical device is placed.
- Fit the Lid onto the Base.
- Stella IQ displays the contact time countdown.
- Upon completion of the cleaning cycle Stella automatically drains Tristel Clean from the Base.

Contents

- Isopropyl Alcohol, Dipropylene Glycol n-Propyl Ether.
- Alkyl (C₁₂₋₁₆) Dimethylbenzyl Ammonium Chloride.

Warning

- Causes skin irritation. Causes serious eye irritation.

Precautionary Statements

- Wear protective gloves/ protective clothing/eye protection.
- **IF ON SKIN:** Wash with plenty of water.
- **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- If skin irritation occurs: Get medical advice/ attention.
- If eye irritation persists: Get medical advice/ attention.

UFI: V110-DoY8-HooW-o6HH

If you believe that this device has contributed to a serious incident (i.e. a death, the temporary or permanent serious deterioration of a person's state of health, or a serious public health threat), contact the device manufacturer and your national competent authority immediately.

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IMPORTERS

EUROPEAN UNION (EU) / EUROPEAN ECONOMIC AREA (EEA) / NORTHERN IRELAND

AUSTRIA, BULGARIA, CROATIA, CZECH REPUBLIC, ESTONIA, GERMANY, GREECE, HUNGARY, LATVIA, LITHUANIA, POLAND, SLOVAKIA, SLOVENIA

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BELGIUM, FRANCE, LUXEMBOURG, NETHERLANDS

Tristel NV/SA

Smallandlaan 14B, 2660 Antwerp, Belgium

T +32 (0) 3 889 26 40 - **E** belgium@tristel.com

W www.tristel.com

IRELAND

Brennan & Co NI

61 Birch Avenue, Stillorgan Industrial Park, Stillorgan, Co. Dublin, A94 XW68, Ireland

T +44 2890 410 136 - **E** info@brennanconi.com

W www.brennanconi.co

ITALY

Tristel Italia Srl

Via Edoardo Collamarini, 14, 40138 Bologna, Italy

T +39 051 0014912 - **E** infoitalia@tristel.com

W www.tristel.com

NORTHERN IRELAND

Brennan & Co NI

Unit 6 & 7, Carrowreagh Business Park, Dundonald, Belfast, Northern Ireland, BT16 1QQ

T +44 2890 410 136 - **E** info@brennanconi.com

W www.brennanconi.com

SPAIN

Vesismin, S.L.

C/ Aribau 230-240, 6^a planta, L-O, 08006 Barcelona, Spain

T +34 934 09 53 01 - **E** vesismmin@vesismmin.com

W www.vesismmin.com

SWITZERLAND

Tristel AG

Sandgrube 29, 9050 Appenzell, Switzerland

T +41 (0)71 5670658 - **E** schweiz@tristel.com - **W** www.tristel.com

Tristel™

WE HAVE CHEMISTRY.

Tristel Solutions Ltd, Lynx Business Park, Fordham Road, Snailwell, CB8 7NY, UK
T +44 (0)1638 721500 - E mail@tristel.com - W www.tristel.com

Tristel NV, Smallandlaan 14B,
2660 Antwerp, Belgium

Tristel AG, Sandgrube 29,
9050 Appenzell, Switzerland

Australian sponsor: Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192, AU
T 1300 680 898 - E mail-au@tristel.com - W www.tristel.com

New Zealand: Tristel New Zealand Ltd, 23 Birch Avenue, Tauranga 3110, NZ
T +64 (0)7 5771560 - E info@tristel.co.nz - W www.tristel.com

Where necessary, external translations have been sourced from ISO 17100:2015
approved supplier: www.tolingo.com.