



# THE WORLD'S MOST WIDELY APPROVED, VALIDATED AND USED HIGH-LEVEL DISINFECTION WIPES SYSTEM FOR LARYNGOSCOPE BLADES

Within emergency care every second counts. Which is why we created the Tristel Trio Wipes System to be quick and mobile, for use in emergency rooms and rescue vehicles.

**Tristel Trio™**  
**Wipes System**

**THE TRISTEL TRIO WIPES SYSTEM IS EASY TO TRANSPORT & USE!**



## CLEAN

### Tristel Pre-Clean Wipe

A non-woven wipe impregnated with a triple enzymatic detergent and surfactant used for a thorough cleaning of the medical device's surface to remove soil and organic matter.



## HIGH-LEVEL DISINFECT

### Tristel Sporidical Wipe

This wipe combines with the Tristel Activator Foam to generate chlorine dioxide, a highly effective biocide that eliminates bacteria, viruses, fungi, mycobacteria and spores from the surface of the medical device in 30 seconds.



## RINSE

### Tristel Rinse Wipe

A sterile packed, non-woven wipe impregnated with deionised water and a low antioxidant content; used to remove any remaining chemical residues from the surface of the medical device.



## TRACE

### Tristel 3T or the Quality Audit Trail Record Book

Intended for the simple registration of all steps of the process, including validation by the responsible person.



## QUICK

The Tristel Trio Wipes System provides a full decontamination cycle, including traceability, in a matter of minutes.



## MOBILE

The Tristel Trio Wipes System is portable and requires no electricity or water to use. This enables it to be easily used in both the emergency room or in rescue vehicles.



## COMPATIBLE

The Tristel Trio Wipes System has been tested and confirmed to be compatible with medical devices from the leading laryngoscope manufacturers, including:

- Airtraq\*
- Medtronic
- Karl Storz
- Verathon

\* Compatible with the Tristel Sporidical Wipe as part of the Tristel Trio Wipes System



## UNRIVALLED EFFICACY

The Tristel Sporidical Wipe is proven effective as a high-level disinfectant according to EN 14885:2018 and the latest efficacy standards published, in 30 seconds. The product's fast-acting and uniform contact time leaves no room for doubt or error.



## SMART TRACEABILITY

The Tristel Trio Wipes System includes either paper-based or digital traceability. The widely-used Quality Audit Trail Record Book documents each disinfection event with pen and paper. Tristel 3T digitalises this process for a more efficient, accurate and smart approach to traceability.




## SIMPLE

Training and certification is an essential part of customer service with Tristel. Our Online Training Portal provides Tristel Trio Wipes System user training at a time and place convenient to you.

The Tristel Trio Wipes System offers complete decontamination of laryngoscope blades. Thanks to the concept of cleaning, high-level disinfection and rinsing, a validated result is achieved with a simple and fast three-step process.



## Microbiological Testing

	Microbes Including	Uniform Contact Time
Bacteria	MRSA, VRE, <i>Klebsiella pneumoniae</i> , <i>Staphylococcus aureus</i> , <i>Pseudomonas aeruginosa</i> , <i>Enterococcus hirae</i>	
Yeasts/ Fungi	<i>Candida albicans and auris</i> , <i>Aspergillus brasiliensis</i>	
Viruses	HPV, HBV, HCV, HIV, Herpes Simplex virus, Norovirus, Adenovirus, Poliovirus	
Mycobacteria	<i>Mycobacterium tuberculosis</i> (TB), <i>Mycobacterium avium</i>	
Spores	<i>Bacillus subtilis</i> , <i>Bacillus cereus</i> , <i>Clostridium sporogenes</i>	

## TRISTEL TRIO WIPES SYSTEM

- Available in 5\* or 50 decontamination procedure packs, with or without the Quality Audit Trail Record Book.

Tristel 3T is sold separately.



## LARYNGOSCOPE BLADE TRANSPORTATION

Tristel protect bags are designed specifically for the transportation and short-term storage (up to 72 hours) of medical devices, with traceability in mind.

### TRISTEL PROTECT

- 50 bags per box.
- Available in small (21cm x 34cm) for laryngoscope blades



\*Subject to market availability. More information about the Tristel Trio Wipes System, such as safety data sheets, reports, publications on materials and studies, is available on request or online at [www.tristel.com](http://www.tristel.com).

**Tristel™**  
WE HAVE CHEMISTRY.

**Manufactured in the United Kingdom by:**  
Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY  
T +44 (0) 1638 721500 · E [mail@tristel.com](mailto:mail@tristel.com) · W [www.tristel.com](http://www.tristel.com)

**Hong Kong & Taiwan:**  
Tristel Asia Limited, 21st Floor, 168 Electric Road, Fortress Hill, Hong Kong  
T +852 2895 6968 · F +852 2869 4388 · E [customerservicehk@tristel.com](mailto:customerservicehk@tristel.com)

**New Zealand:** Tristel New Zealand Limited, 23 Birch Avenue, Judea, Tauranga  
T +64 (0)7 5771560 · F +64 (0)7 5771567 · E [mail-nz@tristel.com](mailto:mail-nz@tristel.com)

**Australia:** Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192  
T 1300 680 898 · F +61 (0)3 9533 6193 · E [mail-au@tristel.com](mailto:mail-au@tristel.com)

For Tristel patent information please visit: <http://www.our-patents.info/tristel>