

# READY TO PREPARE A POD CLO2 10<sup>+5</sup> CAPSULE IN YOUR TANK 10 CLO2 VESSEL? LET'S GET STARTED.



## PRE-USE CHECKS

Some important reminders before starting:

- ✓ Use biocides safely. Always read the product label, safety data sheets (SDS) and product information before use and wear the appropriate personal protective equipment (PPE) when handling disinfectants.
- ✓ Do not use if **POD CLO2 10<sup>+5</sup>** capsules appear damaged or unintentionally open.
- ✓ **POD CLO2 10<sup>+5</sup>** is for healthcare professional use only.
- ✓ Do not apply to brass, copper, aluminium, mild steel, or silver surfaces.
- ✓ Ensure surfaces are free from heavy soiling before use.
- ✓ Once used, **POD CLO2 10<sup>+5</sup>** capsules are not resealable.
- ✓ Consult your vessel user guide prior to preparing your solution for set up and application instructions.
- ✓ Make sure your vessel is empty, it should be located on a flat surface, close to a water source in a central location.
- ✓ Remember to check the tap is closed before filling!
- ✓ To achieve the intended use of **POD CLO2 10<sup>+5</sup>**, please make sure that you follow the provided instructions carefully.

## HOW TO USE



1 Fill vessel with ambient tap water until it reaches the 5L mark. Do not use hot water.



2 Take one **POD CLO2 10<sup>+5</sup>** capsule and place it into the vessel lid with the laminate seal facing down. Close the lid firmly to burst the capsule.



3 Open the lid and remove the now empty capsule. This can be rinsed out and sent for recycling.



4 Fill vessel with additional ambient tap water until it reaches the 10L mark. Close the lid.



5 Allow the solution to dwell for 15 minutes to activate, before it is ready to use.



6 Your working solution expires 5 days after activation within the vessel. To keep track of this, make a note of the time and date on the token in your user guide. Pop out the token and place in the triangular pocket label on the vessel.



7 Grab a **TANK CLO2 Reusable Bottle**, flip the tap to open, and fill with working solution. Each bottle can be paired with the available applicators.



8 On each bottle you decant the solution into, make a note of the time and date with a dry wipe marker. Once decanted from the vessel, your working solution will expire after 24 hours.



9 Apply the working solution onto a dry wipe, or directly onto a surface, and wipe.



10 Ensure the surface remains wet for 5 minutes. Apply additional solution as needed. Leave to dry.

Refer to the User Guide for full instructions.