

Tristel Fuse™ for Surfaces

High-level disinfecting & sporicidal solution for preventing outbreaks and ward closures



1 Pour five litres of ambient water into a container.

Note: Do not use hot water.



2 Take one sachet, fold in half and squeeze one side to burst contents through the centre seal. Contents will start to turn yellow.



3 Tear or cut the sachet.



4 Pour the contents into the five litres of water.



5 Apply disinfectant to surfaces with a mop for floors or a cloth for damp dusting and leave to dry.



6 Ensure a five minute contact time.

Notes:

- Make up solution in a well-ventilated area.
- Wear appropriate PPE. Refer to the Safety Data Sheet and to your hospital protocol.
- Store in original packaging in a cool, well-ventilated area out of direct sunlight.
- Once mixed, solution can be kept for up to 24 hours when stored in an air tight container.
- Dispose of empty packaging in accordance with local policy and national regulations.
- Avoid contact with skin and eyes. Contact with the disinfectant may cause irritation. Wash affected areas with plenty of soap and water.
- Rinse container with water after use.
- Do not use if packaging is damaged.
- For professional use only.

Tristel

Created by: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Tristel order codes:
Fragranced – TSL010101
Unfragranced – TSL010201

NHS order codes:
Fragranced – FAL649
Unfragranced – FAL650

Copyright © Tristel Solutions
Mkt-Wal-113-2