# Tristel Fuse<sup>TM</sup>

for Stella

# High-level disinfecting and sporicidal solution for non-lumened and single-lumened endoscopes, manometry catheters and ultrasound probes

## EACH SACHET CONTAINS

- 50ml Base solution (solution of citric acid with preservatives and corrosion inhibitors in demineralised water)
- 50ml Activator solution (sodium chlorite solution in demineralised water)

This makes 100ml of chlorine dioxide concentrate. Added to five litres of ambient water, each sachet delivers a single-use disinfectant solution for use in the Stella System.

## THE TRISTEL CHEMISTRY

Tristel Fuse for Stella utilises Tristel's proprietary chlorine dioxide chemistry, a well-documented, highly effective biocide. The chemical symbol for chlorine dioxide is ClO<sub>2</sub>.

## MODE OF ACTION

Tristel Fuse for Stella incorporates two separate compartments that contain the Tristel Base and Activator solutions. When mixed by bursting the sachet, chlorine dioxide is generated.

Chlorine dioxide is a powerful oxidising agent – an electron receiver. This means that the chlorine dioxide molecule is in constant search for an additional electron. When a bacterial cell comes into contact with chlorine dioxide, it donates an electron from its cell wall. This creates a breach in the cell wall through which cell contents pass in an attempt to bring the concentrations on either side of the cell membrane to equilibrium. The cell dies through lysis.

## BIOCIDAL PERFORMANCE

Tristel Fuse for Stella is sporicidal, mycobactericidal, virucidal, fungicidal and bactericidal with a contact time of only five minutes. Tristel Fuse for Stella is effective against microorganisms of concern such as:

Spores      *Clostridium sporogenes, Bacillus cereus, Bacillus subtilis*

---

Mycobacteria      *Mycobacterium avium, Mycobacterium terrae*

---

Viruses      Adenovirus Type 1, Poliovirus Type 5, Vaccinia virus,  
Human Papillomavirus (using Polyoma Virus SV40 surrogate), Murine norovirus

---

Fungi      *Aspergillus brasiliensis, Candida albicans, Trichophyton interdigitale*

---

Bacteria      Vancomycin-Resistant Enterococci (VRE) *Enterococcus faecium, Enterococcus hirae,*  
*Escherichia coli, Pseudomonas aeruginosa, Staphylococcus aureus,*  
*Klebsiella pneumoniae*

---


**Manufactured by:** Tristel Solutions Ltd

For Tristel patent information please visit [www.our-patents.info/tristel](http://www.our-patents.info/tristel)

Contact Tristel, your local distributor or visit [www.tristel.com](http://www.tristel.com) for supporting documents such as safety data sheets, microbiological test data and reports.

## HOW TO USE TRISTEL FUSE FOR STELLA


- These Instructions for Use (IFU) should be used in conjunction with the product label, Safety Data Sheet (SDS) and device manufacturer's instructions.
- Ensure pre-clean and cleaning steps are performed after medical device use as per hospital protocol, prior to high-level disinfection with Tristel Fuse for Stella.
- Make up solution in a well ventilated area.
- Wear an apron and nitrile gloves before commencing and wash hands after removing apron and gloves.
- Store in original packaging in a cool, well ventilated area out of direct sunlight.
- Dispose of empty packaging in accordance with local policy and national regulations.
- Avoid contact with skin and eyes. Contact with the disinfectant may cause mild irritation. Wash affected areas with plenty of soap and water.
- For professional use only.

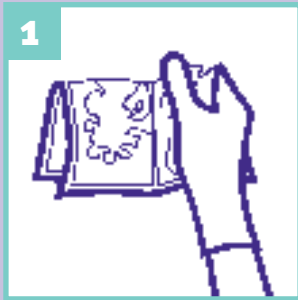
Take one sachet to produce five litres of chlorine dioxide solution. Fold in half and squeeze one side of sachet to burst contents through centre seal.

Contents will start to turn yellow. Allow 30 seconds mixing time.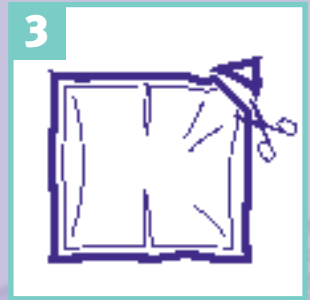

Tear or cut the corner of sachet. Take care when opening the sachet not to spill the concentrated solution.

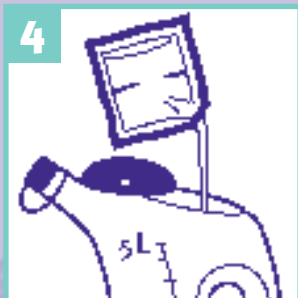Pour contents into five litres of ambient water.

*Note: The mixing of solution should be performed using ambient water. Do not use hot water.*

Add prepared Fuse solution to the Stella System when prompted by Stella IQ. The cycle time is five minutes.

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Tristel Solutions Ltd, Lynx Business Park, Snailwell,