

# LOOKING TO REDUCE PATIENT WAITING TIME AND FREE UP RESOURCES?



**No capital spend, consistent evidence, complete solution – no sourcing of additional products.**

A comprehensive decontamination process for heat-sensitive, non-lumened medical devices such as nasendoscopes, laryngoscope blades, transvaginal and transrectal ultrasound probes, transoesophageal echocardiogram, manometry catheters, ophthalmic devices and intubation endoscopes.

The Tristel Trio Wipes System enables a fast and practical decontamination process at the point of care, eliminating the need to send devices to CSSD.

**Tristel Trio**<sup>™</sup>  
**Wipes System**

## WHY CHOOSE THE TRISTEL TRIO WIPES SYSTEM?



### COMPLIANT

Tristel Trio complies with EN 14885, including the sporicidal norm EN 17126 in 30 seconds.



### EFFECTIVE

Tristel Sporicidal Wipes, part of the Tristel Trio Wipes System are sporicidal, mycobactericidal, virucidal, fungicidal and bactericidal.



### QUICK

Tristel Trio provides a full decontamination process, including traceability, in a matter of minutes.



### COMPATIBLE

Tristel Trio is tested and approved by major manufacturers.



### MOBILE

Tristel Trio can be used at the point of care, without the limitations machine-based systems bring.



### ECONOMICAL

Tristel Trio eliminates investment in machine-based systems and maintenance contracts.

## THE TRISTEL TRIO WIPES SYSTEM IS EASY TO USE!



### CLEAN

#### Tristel Pre-Clean Wipe

A non-woven wipe impregnated with a triple enzymatic detergent and surfactant. The wipe is used for thorough cleaning of the medical device's surface to remove soil and organic matter.



### HIGH-LEVEL DISINFECT

#### Tristel Sporicidal Wipe

This wipe combined with the Tristel Activator Foam generates chlorine dioxide, a highly effective biocide that delivers high-level disinfection in a uniform contact time of 30 seconds.



### RINSE

#### Tristel Rinse Wipe

A sterile packed, non-woven wipe impregnated with deionised water and low antioxidant content. The wipe is used to remove any remaining chemical residues from the surface of the medical device.



### TRACE

#### The Quality Audit Trail Record Book or 3T (NEW)


Two traceability options for the simple registration of all steps of the process, including validation by the responsible person.


## POWERED BY TRISTEL CHLORINE DIOXIDE (ClO<sub>2</sub>)

The Tristel Sporicidal Wipe utilises Tristel's proprietary chlorine dioxide chemistry (ClO<sub>2</sub>), a well-documented and highly effective biocide.

ClO<sub>2</sub> is a powerful oxidizing agent that packs a punch. It destroys pathogens via electron exchange, sequestering electrons from the microorganism's vital structures such as cell walls, membranes, organelles, and genetic materials. This causes a molecular imbalance leading to the microorganism's death.

This mode of action makes ClO<sub>2</sub> highly efficacious, even at low concentrations.

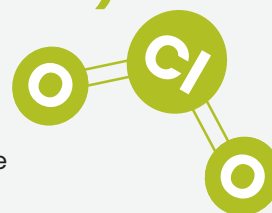

	European norms (EN)	Uniform Contact Time
Bacteria	EN 13727	
	EN 14561	
	EN 16615	
Yeasts/Fungi	EN 13624	
	EN 14562	
	EN 16615	
Viruses	EN 14476	
Mycobacteria	EN 14348	
	EN 14563	
Spores	EN 17126	
Did you know?		
<p>EN 17126 is the first standard for the evaluation of sporicidal activity in the medical area, and compliance with this test standard is mandatory to claim sporicidal efficacy. The previous standard EN 13704 is no longer valid and applied up until the release of EN 17126 in 2018.</p> <p><b>Does your current system comply with EN 17126?</b></p>		

**The Tristel Trio Wipes System** is the most widely approved, validated and used high-level disinfection wipes system globally. Since its launch, Tristel Trio has protected patients in more than

**100million**  
procedures worldwide.

**NEW**

## SMART TRACEABILITY

The Tristel Trio Wipes System includes either paper-based or digital traceability.

The widely-used Quality Audit Trail Record Book documents each disinfection event with pen and paper. Tristel 3T digitalises this process for a more efficient, accurate and smart approach to traceability.





## NOT CONVINCED YET?

Try TRISTEL TRIO as your reliable back-up solution for decontamination.

You can count on Tristel Trio:

when your machine breaks down

when you are short of equipment

when you want to reduce patient waiting times

Tristel Trio brings reliability to your practice, keeping it operational, practical and to the best standard at all times.



## ORDER DETAILS AND SUPPORTING PRODUCTS

### TRISTEL TRIO WIPES SYSTEM

- Available in a box of 50 decontamination procedures, with or without the Quality Audit Trail Record Book.
- Product code: TSL030601.


### TRISTEL 3T

- Available on request.



More information about the Tristel Trio Wipes System, such as safety data sheets, reports, publications on materials and studies, is available on request or online at [www.tristel.com](http://www.tristel.com).

**Tristel**™

 Refer to the User Guide for full product instructions.

**Australia:**  
Tristel Pty Ltd,  
Unit 44, 328 Reserve Road,  
Cheltenham, VIC 3192, Australia  
T 1300 680 898  
E [mail-au@tristel.com](mailto:mail-au@tristel.com)  
W [www.tristel.com/au-en/](http://www.tristel.com/au-en/)



Scan for more

**New Zealand:**  
Tristel New Zealand Ltd,  
23 Birch Avenue, Judea,  
Tauranga 3110  
T +64 (0) 7 5771560  
E [mail-nz@tristel.com](mailto:mail-nz@tristel.com)  
W [www.tristel.com/nz-en/](http://www.tristel.com/nz-en/)



Scan for more

Copyright © Tristel Solutions Limited  
Mkt-Bro-2737 - 17/DEC/2025 - V2

For Tristel patent information please visit:  
<http://www.our-patents.info/tristel>