

NOTES

- **Do not** use Tristel DUO OPH if the bottle is damaged or unintentionally opened.
- **Disinfect hands** and **wear suitable gloves** and Personal Protective Equipment (PPE) when handling disinfectants and medical devices.
- Clean medical devices thoroughly prior to disinfection. Residual contamination with organic soiling will decrease the effectiveness of any disinfectant. Follow hospital protocol and medical device manufacturer's instructions.
- **Do not** use on soft metals or low grades of stainless steel.
- Close the Tristel DRY WIPES cap after each use, do not refill the tub or use it for other purposes.
- This wall chart should be used in conjunction with the instructions for use (IFU), product label, Safety Data Sheet (SDS) and device manufacturer's instructions.
- For healthcare professional use only.


DIRECTIONS FOR USE




Clean your hands and apply clean gloves, then dispense **two doses** of Tristel DUO OPH onto a Tristel DRY WIPE.



Smaller Devices: For smaller devices, lay the wipe in the palm of your hand and apply two doses of Tristel DUO OPH. Place the device directly into the foam and use the wipe to ensure all areas of the device are covered.

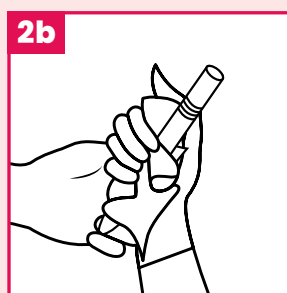


Larger devices: Use the wipe to spread the foam over the surface of the medical device and ensure that all areas are covered. Pay particular attention to any crevices, ridges, or indentations.

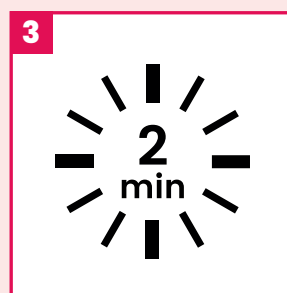


Ensure the device remains wet for a minimum contact time of **2 minutes**. Discard the used wipe and gloves in accordance with local regulations. Do not re-use.



Clean your hands and put on new gloves.

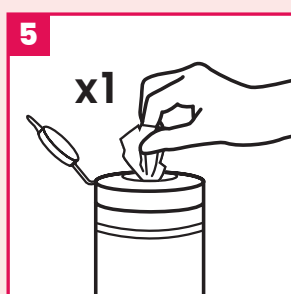

Pull **one** Tristel DRY WIPE out of the tub and close the cap tightly. **Wipe the device once over** to remove residues and dry it. Pay close attention to indentations and joins along the device.

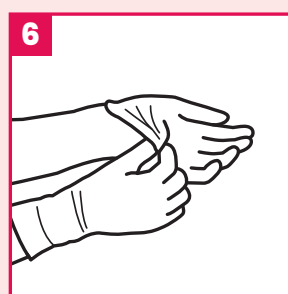

Discard the used wipe and gloves in accordance with local regulations. Do not re-use. Store the medical devices in accordance with hospital protocols to prevent damage or recontamination.