





SPORICIDAL DISINFECTION THAT KEEPS ON PROTECTING

JET PRO is a sporicidal chlorine dioxide foam with an extra ingredient that enables the foam to effectively eliminate bacteria and enveloped viruses for up to 8 hours after application.


Disinfectants cannot be used every minute of every day. The combination of **JET PRO's** powerful chlorine dioxide chemistry with prolonged protection technology provides assurance for patients, staff and visitors, even after you have moved on to your next task.

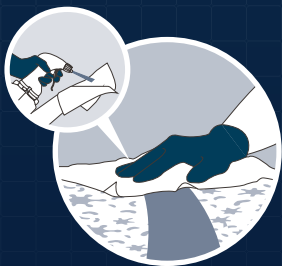


JET PRO is sporicidal in 1 minute.

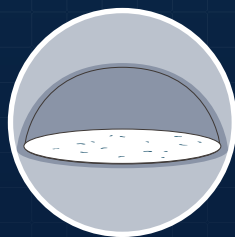

Once dry, **JET PRO** places an invisible barrier on the surface, protecting the surface for up to 8 hours.

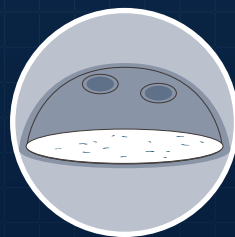

Touching the surface weakens the protective barrier, and regrowth of microorganisms will eventually occur.

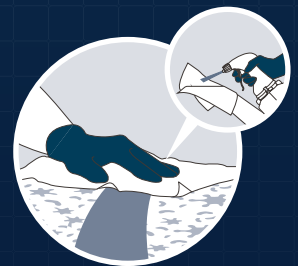

JET PRO can then be re-applied to remove microorganisms and refresh the protective barrier.

WET KILL

DRY KILL

WET KILL

TRISTEL CHLORINE DIOXIDE (ClO₂) + PROLONGED PROTECTION TECHNOLOGY

JET PRO is built upon the same chlorine dioxide (ClO₂) chemistry as the Tristel Medical Device Decontamination products. **JET PRO** incorporates two separate compartments that contain Base Solution (citric acid) and Activator Solution (sodium chlorite).

When mixed by pulling the **JET PRO** trigger, ClO₂ chemistry is generated.

JET PRO contains an amphoteric surfactant for added cleaning performance.

ClO₂ eliminates microorganisms by destroying their molecular structures.

This means that harmful microbes cannot build resistance to ClO₂. An extra ingredient enables the foam to effectively eliminate bacteria and enveloped viruses for up to 8 hours after application.



SAY GOODBYE TO PRE-WETTED PLASTIC WIPES

Efficacy with the standard wipe used in EN 16615 means **JET PRO** can be paired with any dry wipe.

Use a microfibre cloth or a plastic free dry wipe to create sporicidal wipes.

MICROORGANISM TYPE	MICROORGANISM	TEST METHOD
Bacterial Spore	<i>Clostriduisudes difficile</i>	EN 17126
Mycobacteria	<i>Mycobacterium terrae, Mycobacterium avium</i>	EN 14563
Virus	Herpes Simplex Virus Type 1, Poliovirus Type 1, Adenovirus Type 5	ASTM E 1053
Fungi/Yeast	<i>Aspergillus basiliensis, Candida albicans</i>	EN 14562
		EN 13624
Bacteria	<i>Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae</i>	EN 14561
		EN 13727
RESIDUAL ACTIVITY		
Virus	<i>Vaccinia virus</i>	PAS 2424
Bacteria	<i>Pseudomonas aeruginosa, Staphylococcus aureus Enterococcus hirae, Escherichia coli</i>	PAS 2424



DID YOU KNOW?

PAS 2424 is the only test method available to evaluate a disinfectant's long-last antimicrobial activity.

EASY TO USE, APPLY, WIPE & WAIT

Each bottle of **JET PRO** delivers 570 doses of sporicidal chlorine dioxide foam.

Simply apply, wipe and wait!

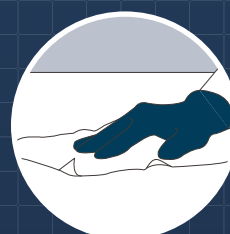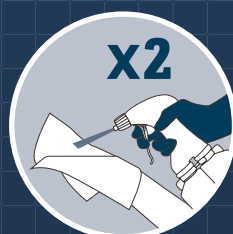

Contact your local Sales Representative to learn how **JET PRO** can improve your infection control procedures.

Tristel Surfaces™

Tristel New Zealand

23 Birch Avenue, Judea, Tauranga, Bay Of Plenty, 3110

T +64 (0) 7 5771560 E mail-nz@tristel.com W www.tristel.com/nz-en/

Copyright © Tristel Solutions Limited
Mkt-Bro-3725 - 30/JUL/2026 - V1

*Modified for virucidal efficacy against enveloped viruses. Use biocides safely. Always read the label and product information before use.