

A TRIPLE-ENZYME FOAM FOR MEDICAL DEVICE CLEANING

Tristel Clean is designed specifically for use on non-lumened invasive and non-invasive medical devices.

Tristel Clean can be used as a stand-alone cleaning product, or prior to disinfection.

Disinfection is only effective when a device has undergone thorough cleaning.

No rinsing is required after using Tristel Clean on its own, or before Tristel disinfectants.



Tristel[™]
Clean

TRISTEL CLEAN IS MOBILE, PRACTICAL & COMPATIBLE



TRIPLE ENZYME
MEETS CLEANING STANDARDS



LONG LASTING
TWO-YEAR SHELF LIFE



MOBILE
LIGHT-WEIGHT & PRACTICAL



COMPATIBLE
APPROVED BY MAJOR
MANUFACTURERS



PRACTICAL
NO SOAKING REQUIRED



FOAM
NON-AEROSOL APPLICATION

**TRISTEL CLEAN PASSES LOCAL REGULATORY REQUIREMENTS
FOR AUSTRALIA & NEW ZEALAND**

TRISTEL CLEAN IS EASY TO USE. SIMPLY APPLY & WIPE.



**DISPENSE TRISTEL CLEAN
ONTO A DRY WIPE.**

Duo Wipes are recommended.



**SPREAD OVER DEVICE UNTIL SOIL
AND ORGANIC MATTER HAVE BEEN
VISIBLY REMOVED.**

In case of heavy soiling repeat the process.



**IF REQUIRED, DISINFECT THE
DEVICE IN LINE WITH DEVICE
APPLICATION REQUIREMENTS.**

Refer to the Tristel disinfectant user guide
for full instructions.

APPROVED BY MANUFACTURERS AND REGIONAL REGULATORY BODIES

Tristel Clean is designed for use on medical devices prior to disinfection with Tristel Duo, Tristel Fuse for Stella or the Tristel Trio Wipes System. It can also be used as a stand-alone cleaning product for procedures such as non-critical ultrasounds.

The Tristel Clean formula has been used for over twenty years in the Tristel Pre-Clean Wipe and is approved for use by major manufacturers such as:

- Advanced Tactile Imaging, Inc.
- Arthrex
- ATMOS MedizinTechnik
- Becton Dickinson (Bard Access, CareFusion)
- BK Medical
- Butterfly Network
- Canon Medical Systems
- Carestream
- Clarius Mobile Health
- Clarus Medical
- DWL Compumedics
- Echosens
- Exact Imaging
- Fisher & Paykel
- Fujifilm (Fujinon)
- Fujifilm (Hitachi)
- FUJIFILM SonoSite
- GE Healthcare
- Haag-Streit
- Karl Storz
- Laborie (Cogentix Medical, MMS)
- Machida Endoscope Co.
- MCube
- Medcom
- Mindray
- Neoplas Tools
- Ocular Instruments Inc
- Optim LLC
- Orvision
- Pentax
- Philips
- Polydiagnost
- Samsung Healthcare
- SapiMed
- Siemens Healthineers
- Southern Scientific / Care Wise
- tic Medizintechnik
- Unisensor
- Verathon

DUO WIPES

200 wipes per tub / 6 tubs per box



TRISTEL CLEAN

310 aliquots per bottle / 6 bottles per box

Tristel™

Medical Device
Decontamination

UK Manufacturer:

Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

New Zealand:

Tristel New Zealand Limited, 23 Birch Avenue, Judea, Tauranga
T +64 (0)7 5771560 - F +64 (0)7 5771567 - E mail-nz@tristel.com

Australia:

Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192
T 1300 680 898 - F +61 (0)3 9533 6193 - E mail-au@tristel.com

For Tristel patent information please visit: <http://www.our-patents.info/tristel>
Copyright © Tristel Solutions - Mkt Bio-1344-2 - 18/MAR/2022

Tristel™
WE HAVE CHEMISTRY.