


Tristel Duo™

NCU

FOR NON-INVASIVE
ULTRASOUND PROBES

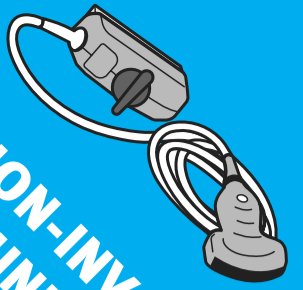

Intermediate Level Disinfectant

Notes:

- Do not use if the bottle is damaged.
- Disinfect hands and wear appropriate Personal Protective Equipment (PPE) when handling disinfectants and medical devices.
- Remove and discard transport locks that stop the pump from being depressed in transit.
- If Tristel Duo NCU is used for the first time, depress the pump two to four times to prime it.
- Not to be used on soft metals such as brass or copper.
- For professional use.

STEP 1



Depress the pump to dispense Tristel Duo onto a dry wipe (Duo Wipe is recommended) or directly onto the surface.

STEP 2



Use the wipe to spread the foam over the surface and ensure all areas are wet.

STEP 3



Allow the surface to remain wet for 30 seconds. Leave to air dry.

Note: Discard the used wipe to clinical waste. Do not macerate. Do not re-use.

Tristel™

Created by: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

Australian sponsor: Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192
T 1300 680 898 - F +61 (0)3 9533 6193 - E mail-au@tristel.com

New Zealand sponsor: Tristel New Zealand Limited, P.O Box 15451, Tauranga, New Zealand
T +64 (0) 7 5771560 - E info@tristel.co.nz

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Copyright © Tristel Solutions
Mkt-Wal-508-1
May 2018