



### Reusable Hygienic Tourniquet

#### STEP 1



#### APPLICATION

Loop daisygrip around the patient's limb, click magnetic clasp together.

#### STEP 2



#### TIGHTENING

Place finger between strap and skin to prevent pinching. Pull strap from close to the clasp at a 45° angle to tighten – do not pull from the distal (tail end) of the strap.

#### STEP 3



#### LINE INSERTION/BLOOD COLLECTION

Loosen the daisygrip once the needle is in the vein.


#### STEP 4



#### LOOSENING

Place finger under clasp. Use a 'lifting' motion to flip the clasp in the direction of the strap to loosen daisygrip and release stasis.

#### STEP 5

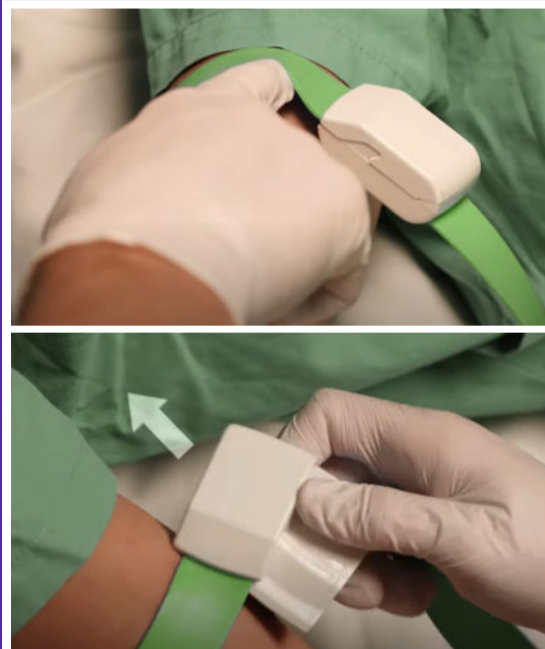

#### REMOVING

Loosen, then slide the top clasp sideways with your thumb, whilst holding the bottom arm of the clasp steady (with fingers or palm).

#### STEP 6



#### REPROCESSING

Ensure clasp and strap are cleaned and/or disinfected in line with local requirements after every use. Strap should be dry before next use.