DISTEL

c

DOSING SYSTEM

cache[™]

DISTEL DOSING SYSTEM is intended for the cleaning and disinfection of surfaces of non-invasive medical devices and equipment, and of hard non-porous environmental surfaces in hospitals, care facilities, GP surgeries, laboratories, emergency vehicles, dentist practices, manufacturing facilities, hospitality venues, educational facilities and veterinary care. It is not intended for the disinfection of instruments and invasive medical devices.

DISTEL DOSING SYSTEM denatures DNA and RNA, and is effective against bacteria, yeast, and viruses.

DISTEL DOSING SYSTEM is marketed in compliance with the EU Medical Devices Directive(93/42/EEC and Amendment 2007/47/EC thereto), EU Biocidal Products Regulation ((EU) 528/2012) and/or local transitional laws for biocidal products.

Active: DIDECYL DIMETHYL AMMONIUM CHLORIDE <0.35g/kg.

INSTRUCTIONS FOR USE

- For professional use only.
- Wear appropriate Personal Protective Equipment (PPE).
- Remove any solid waste from the surface prior to use.
- Do not use on electrical components.
- Dispose of in accordance with local policy and national regulations.
- Not to be used on soft furnishings or porous materials.

PREPARING THE WORKING SOLUTION

- Check solution expiry date before use, located on the concentrate bottle label.
- Before preparing the working solution, record the concentrate LOT and Expiry onto the dosing bottle label with a dry wipe pen for solution traceability. Dispose of after 6 months.
- If the expiry of the Concentrate is before six months, state Concentrate expiry date as the Working Solution expiry date.

1 LITRE DISTEL DOSING SYSTEM


Fill with 100ml of concentrate solution up to the first volume marker on the dosing bottle.

Fill with 900ml of ambient tap water until solution reaches the top volume marker on the dosing bottle.

500ml DISTEL DOSING SYSTEM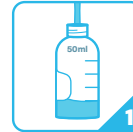

Fill with 50ml of concentrate solution up to the first volume marker on the dosing bottle label.

Fill with 450ml of ambient tap water until solution reaches the top volume marker on the dosing bottle.

Affix the Doser to the dosing bottle.

Choose from any of the available applicators, and affix to the dosing bottle.

INSTRUCTIONS FOR USE

THE DOSER

Ensuring the cap is closed, invert the bottle 180 degrees to fill the Doser. Return the bottle to an upright position.

Open the cap and dose the liquid solution directly onto a surface, or onto a wipe of your choice, enough to fully saturate.

Wipe the surface, ensuring it remains wet for the required contact time (see Table 1).

THE SPRAYER

Spray the liquid solution directly onto a surface, or onto a wipe of your choice, enough to fully saturate.

Wipe the surface, ensuring it remains wet for the required contact time (see Table 1).

THE FOAMER

Remove cap and, press the foamer to apply directly onto a surface, or onto a wipe of your choice, enough to fully saturate. Replace cap.

Wipe the surface, ensuring it remains wet for the required contact time (see Table 1).

Note: If the foamer is being used for the first time, depress it two to four times to prime it.

REPURPOSING THE BOTTLE

Once the concentrate solution has been depleted, the concentrate solution bottle can be repurposed into a dosing bottle for working solution.

- Before reusing the dosing bottle, remove all traceability data from the label.
- Rinse the dosing bottles with tap water between uses.

Rinse out the dosing bottle with tap water.

Remove the concentrate label from the bottle, and dispose to general waste.

Apply the working solution label provided onto the dosing bottle.

This bottle can now be used to transport and dispense working solution.

ORGANISM	TEST NORM	MICROORGANISM	CONTACT TIME	CONDITIONS
VIRUS	EN 14476 Phase 2, Step 1	Feline coronavirus (surrogate of SARS-CoV-2)	2 min.	Dirty 1
	DVV / RKI	Bovine Viral Diarrhea Virus (surrogate of Hepatitis C Virus)	30 sec.	Dirty 2
		Vaccinia virus	3 min.	
YEAST	EN 13624 Phase 2, Step 1	<i>Candida albicans</i>	2 min.	Clean
	EN 16615 Phase 2, Step 2	<i>Candida albicans</i>	5 min.	Clean.
BACTERIA	EN 16615 Phase 2, Step 2	<i>Pseudomonas aeruginosa</i> <i>Staphylococcus aureus</i> <i>Enterococcus hirae</i>	5 min.	Clean
	EN 13727 Phase 2, Step 1	<i>Staphylococcus aureus</i> <i>Enterococcus hirae</i> <i>Pseudomonas aeruginosa</i> <i>Proteus vulgaris</i> , <i>Escherichia coli</i> , Methicillin-resistant <i>Staphylococcus aureus</i> (MRSA), Carbapenem-resistant <i>Enterobacteriaceae</i> (CRE) <i>Klebsiella</i> , Vancomycin-resistant <i>Enterococci</i> (VRE) <i>Enterococcus faecium</i> , Multi drug-resistant <i>Acinetobacter baumannii</i> (MDRAB)	2 min.	Clean

NUCLEIC ACID	METHOD	CONTACT TIME	CONDITIONS
Deoxyribonucleic acid (DNA)	Spectrophotometric and fluorometric	1 min.	Not Applicable
Ribonucleic acid (RNA)			

Key:

Clean: 0.3g/l bovine albumin
 Dirty 1: 5% fetal bovine serum
 Dirty 2: 10% fetal bovine serum

Manufactured by: Tristel Solutions Ltd

For Tristel patent information please visit www.our-patents.info/tristel. Contact Tristel, your local distributor or visit www.tristel.com for supporting documents such as safety data sheets, microbiological test data and reports.

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Fabriqué par: Tristel Solutions Ltd
Suisse: Tristel AG, Sandgrube 29, CH-9050, Appenzell - T +41 715670658 - E schweiz@tristel.com
Tristel Solutions Ltd, Lynx Business Park,