

# TANK

The most economical and effective high-level disinfectant system on the market.



# A new dosing system for SWIFT™


TANK has a capacity of 10 litres, is made of polyethylene, has a footprint of 30 x 30cm and weighs 1.7kg when empty. When prepared in TANK, SWIFT becomes the most economic, most effective and fastest high-level disinfectant on the market.

## HOW TO USE SWIFT WITH TANK



### Step 1


Take two SWIFT sachets and fold each one in half. Squeeze one side to burst contents through the centre seal. Allow a two minute dwell time. Contents will start to turn yellow.

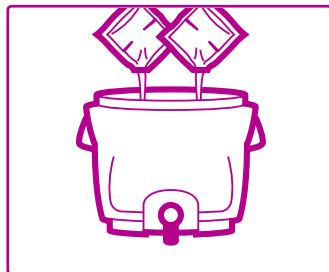

### Step 2

Pour five litres of ambient water into TANK and pour in the contents of both SWIFT sachets. Fill TANK with more ambient water until the 10 litre marking is reached. Close the lid.


**Note:** Do not use hot water.

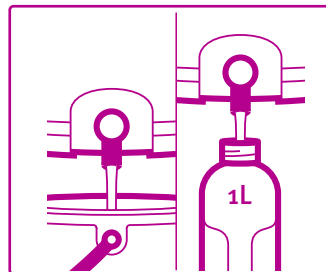

### Step 3

Use the tap on the front of TANK to decant SWIFT working solution into a bucket or into 1L dosing bottles.

**Note:** When prepared in TANK, the shelf life of SWIFT working solution is extended to five days.

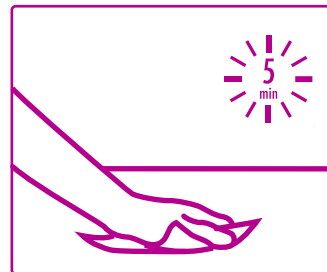

### Step 4

Apply SWIFT to surfaces with a mop for floors or a cloth for damp dusting. Ensure a five minute contact time and leave to dry.

## USING THE DOSING BOTTLES



SWIFT can be decanted into 1L dosing bottles for application onto smaller surface areas. These innovative bottles allow SWIFT working solution to be dosed accurately and cleanly onto a cloth in a single hand movement.

The dose size is delivered with the same precision for each application: the bottle cap has a capacity of 10ml and is pre-dosed when tilting the bottle. This new way of dosing is environmentally friendly and prevents the overuse of disinfectant.

SWIFT is designed for rapid disinfection within:

- Operating theatres
- Consultation rooms
- Isolation rooms
- Unoccupied animal habitats
- Reception areas

**Note:** Swift is not suitable for use on fabric or soft furnishings.

“Hello!  
Follow us on Twitter via  
@TristelGlobal”



## THE BENEFITS OF SWIFT

### ECONOMIC

When prepared in TANK, the shelf life of SWIFT working solution is extended to five days making it the most economic, most effective and fastest high-level disinfectant on the market.

### FAST

SWIFT has a contact time of five minutes.

### EFFECTIVE

Swift is effective against microorganisms including Feline calicivirus, Canine parvovirus, Bovine enterovirus, *Mycobacterium bovis*, *Escherichia coli*, *Salmonella enterica*, *Streptococcus equi*, Foot and Mouth Disease Virus (FMDV) and Swine Vesicular Disease Virus (SVDV).

**SWIFT is approved under DEFRA for FMDV and SVDV.**

## PRODUCT OPTIONS



TANK  
TR027



SWIFT  
(40 Citrus or Cherry Sachets/box)  
TR116



1L Dosing  
Bottle  
TR028

Use biocides safely. Always read the label and product information before use.

**Tristel™**

Created by: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY  
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Hong Kong & Taiwan: 21st Floor, 168 Electric Road, Fortress Hill, Hong Kong  
T +852 2895 6968 - F +852 2869 4388 - E customerservice@tristel.com.hk

Copyright © Tristel Solutions  
Mkt-Fac-095-8  
January 2020