



HOW TO
UPDATE
THE 3T APP

DIGITAL TRACEABILITY SYSTEM DESIGNED TO TRAIN,
TRACK AND TRACE EFFICIENTLY, SECURELY AND ACCURATELY



TURN ON
YOUR 3T DEVICE



CONNECT
TO WIFI



TAP
'APP CATALOG'
=>'TRISTEL APP'
=>'UPDATE'



WAIT
FOR A COUPLE OF MINUTES
WHILST THE UPDATE IS
DOWNLOADING



COMPLETE
THE UPDATE BY TAPPING
NEXT/OK WHEN ASKED TO
INSTALL



INPUT
YOUR APP INSTALLATION
CODE TO START USING
THE UPDATED 3T APP*

*If unsure, please contact your organisation's Super User or Tristel to recover your App Installation Code.



For any enquiries, please contact your local Tristel representative or email 3TSupport@tristel.com

United Kingdom: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - W www.tristel.com

Ireland: Brennan & Company
61 Birch Avenue, Stillorgan Industrial Park, Stillorgan, Co. Dublin, A94 XW68
T 01 2952501 - E enquiries@brennanco.ie - W www.brennanco.ie

Australia: Tristel Pty Ltd.
40/328 Reserve Road, Cheltenham, VIC 3192
T 1300 680 898 - F +61 (0)3 9533 6193 - E mail-au@tristel.com

Hong Kong: Tristel Asia Ltd.
21st Floor, 168 Electric Road, Fortress Hill, Hong Kong
T +852 2895 6968 - F +852 2869 4388 - E customerservice@tristel.com.hk

New Zealand: Tristel New Zealand Ltd.
23 Birch Avenue, Judea, Tauranga, P.O. Box 3110
T +64 (0)7 5771560 - F +64 (0)7 5771567 - E info@tristel.co.nz