

How to use SWIFT™ with TANK

High-level disinfectant for surfaces



- Pour five litres of ambient water into TANK.
Note: Do not use hot water.




- Take two SWIFT sachets and fold each one in half. Squeeze one side to burst contents through centre seal. Contents will start to turn yellow.
- Allow a two minute dwell time. SWIFT is now ready to use, tear or cut the sachet.



- Pour the sachet contents into the five litres of water in TANK. Hold sachets below the water line to avoid splashing the concentrate.
- Fill TANK with more water until the 10 litre marking is reached.



- Screw close the TANK lid, ensuring it is fully tightened. Once mixed, the solution can be kept in TANK for up to **five days**.
- Complete the label with the solution data.

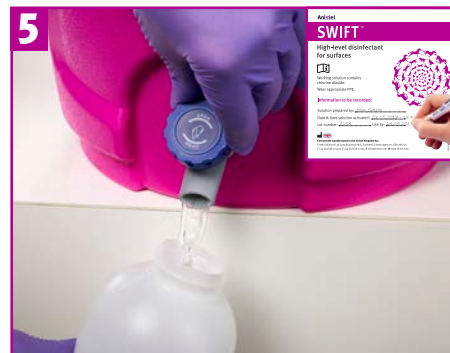

- Use the tap on the front of TANK to decant SWIFT working solution into 1L dosing bottles. The solution can be kept in the dosing bottles for up to **24 hours**.
- Complete the label with the solution data.



- Apply SWIFT to surfaces with a cloth for damp dusting.
- Leave surfaces to dry naturally to ensure a five minute contact time.

Notes:

- Make up solution in a well-ventilated area.
- Wear gloves and apron before commencing and wash hands after removing gloves and apron.
- Avoid contact with skin and eyes. Contact with the disinfectant may cause mild irritation. Wash affected areas with plenty of soap and water.
- In case of contact with eyes, rinse immediately with water and seek medical advice.
- Store in original packaging in a cool, well-ventilated area out of direct sunlight.
- Keep between 10°C and 35°C.
- Do not use on soft furnishings or fabrics.
- Dispose of empty packaging in accordance with local policy and national regulations.
- Keep out of reach of children.
- TANK should be placed on a non-slip surface and remain stationary when in use.
- Rinse and dry TANK and dosing bottles between uses.
- Do not use if packaging is damaged.
- Do not place objects on top of TANK when in use.
- For professional use only.
- Wipe the labels clean to record new data (for fresh solution preparation and dosing bottle refills).

Tristel

Created by: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Copyright © Tristel Solutions
Mkt-Wal-243-1