


TristelTM Solo

for Ultrasound


Disinfectant wipes for skin surface probes and ultrasound stations



Take one Tristel Solo for Ultrasound disinfectant wipe.

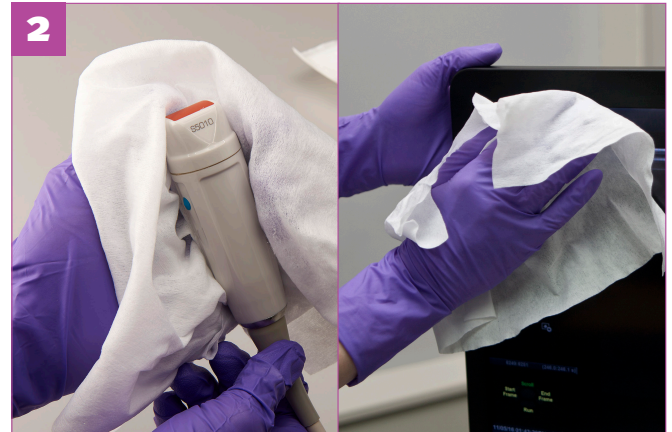

Use the wipe to disinfect the skin surface probe. Use another wipe to disinfect the monitor, keyboard, probe holders and cables.

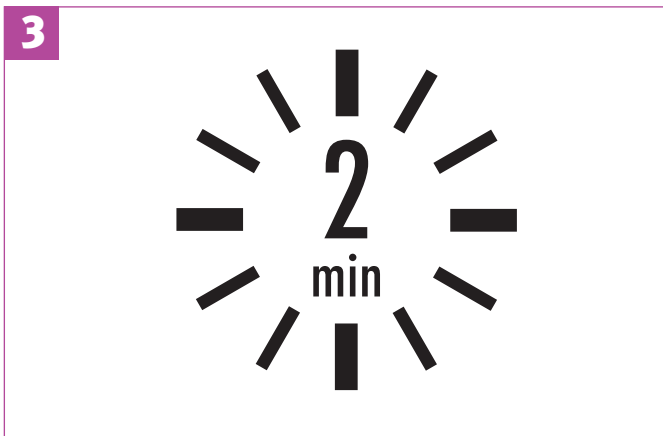

Leave surfaces to dry to ensure a minimum contact time of two minutes.

Notes:

- Disinfect hands and wear appropriate PPE when handling disinfectants and medical devices.
- Discard the wipes and gloves in accordance with hospital guidelines.
- Always follow hospital protocols and manufacturer's instructions.
- Do not use if packaging is damaged.
- For professional use only.

Tristel

Created by: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Copyright © 2016 Tristel Solutions
Mkt-Wal-258-1