

Tristel Trio™ Wipes System

A complete decontamination solution for ophthalmic devices.

FOR DIAGNOSTIC
CONTACT LENSES




Refer to the User Guide for full instructions.

Notes:

- Always follow hospital protocols and device manufacturers' instructions.
- Disinfect hands and wear appropriate PPE when handling disinfectants and medical devices.
- Prepare for the decontamination process by taking one of each Wipe and the Activator Foam from the box.
- Do not use past the expiry date. For the expiry date and LOT number please see the surface of the carton.
- Do not use if the Wipe sachet is damaged, or the Wipe is discoloured, damaged or dry.
- Use more Wipes if required to ensure the instrument is visibly clean.
- Keep the Wipe sachets to complete the Tristel Traceability book at the end of the decontamination procedure.
- Discard the used Wipe and gloves in accordance with local regulations.
- For professional use only.

STEP 1: CLEANING

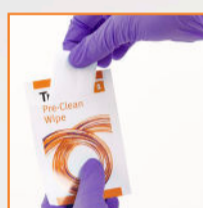

Remove the Pre-Clean Wipe from its sachet and unfold it in the palm of your hand.

Wipe the lens to remove all visible soil.

Discard the used Wipe and gloves to clinical waste.

Disinfect your hands and put on new gloves.

STEP 2a + 2b: ACTIVATING



Remove the Sporicidal Wipe from its sachet and unfold it in the palm of your hand.

Take the Activator Foam bottle and apply two doses of foam onto the Sporicidal Wipe.

Fold the Sporicidal Wipe in on itself and scrunch it during **15 seconds** to generate chlorine dioxide.

Ensure that the Wipe is completely covered in Foam.

Use the activated Wipe immediately.

STEP 2: HIGH-LEVEL DISINFECTING



Wipe the surface of the lens to cover it with foam, ensuring all areas come into contact with the Wipe.

Discard the used Wipe to clinical waste.

Observe a minimum contact time of **30 seconds**.

STEP 3: RINSING



Remove the Rinse Wipe from its sachet and unfold it in the palm of your hand.

Wipe the lens to remove excess foam.

Discard the used Wipe to clinical waste.

STEP 4: TRACEABILITY



Refer to the used Wipe sachets and Activator Foam bottle to complete the Tristel Quality Audit Trail Record Book.

Tristel™

Created by: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0)1638 721500 - E mail@tristel.com - W www.tristel.com

For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Copyright © Tristel Solutions
Mkt-Wal-1603-1
May 2021