


Tristel Duo™

ULT



DESTROY HPV TYPES 16 & 18 IN 30 SECONDS WITH TRISTEL DUO ULT*

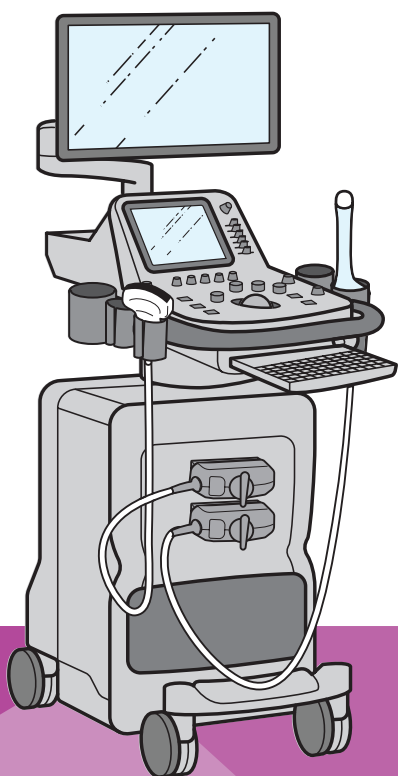

There are over 200 different Human Papillomavirus (HPV) types. Types 16 and 18 are known to be responsible for approximately 70% of cervical cancer cases. A new scientific study by Meyers, et al. (2020) demonstrates that Tristel Duo ULT possesses efficacy against infectious HPV types 16 and 18 on a transvaginal ultrasound probe in 30 seconds.

Tristel Duo ULT is the only high-level disinfectant proven effective against HPV in 30 seconds, designed to treat the entire ultrasound station.

Tristel Duo ULT provides high-level disinfection for endocavity ultrasound probes, skin surface ultrasound probes (including those used during invasive procedures), probe holders, cords, plugs, keyboards, monitors and even mobile medical devices that combine an ultrasound probe with a smart device.

EFFECTIVE AGAINST ORGANISMS OF CONCERN IN OBSTETRICS & GYNAECOLOGY

Tristel Duo ULT is sporicidal, mycobactericidal, virucidal, fungicidal and bactericidal in a contact time of 30 seconds. Quick and effective high-level disinfection with Tristel Duo ULT is vital in the prevention of infections within obstetrics and gynaecology such as:



SKIN SURFACE GUIDED PROCEDURE INFECTIONS

CAUSED BY *STAPHYLOCOCCUS AUREUS* & *PSEUDOMONAS AERUGINOSA*



SEXUALLY TRANSMITTED DISEASES

CAUSED BY HPV, HIV, & *CHLAMYDIA TRACHOMATIS*



RISK OF CORONAVIRUS* TRANSMISSION

OF PARTICULAR CONCERN IN PREGNANT PATIENTS



BACTERIA & YEAST VAGINITIS

CAUSED BY *GARDNERELLA VAGINALIS* (BACTERIA) & *CANDIDA ALBICANS* (YEAST)



INCREASED RISK OF FAECAL CONTAMINATION

CAUSED BY *ESCHERICHIA COLI*

COMPLIANT

Tristel Duo ULT complies to EN 14885:2018 and the latest applicable EU standards published

EFFECTIVE

Tristel Duo ULT is sporicidal, mycobactericidal, virucidal, fungicidal and bactericidal

QUICK

Tristel Duo ULT is effective in a contact time of just 30 seconds

COMPATIBLE

Tested and confirmed to be compatible by major manufacturers

MOBILE

Tristel Duo ULT can be used wherever required without the limitations automated systems bring

CONVENIENT

Training can be undertaken via the Tristel Online Training Portal at your convenience



TRISTEL DUO ULT

DESTROYS HPV ON REAL LIFE DEVICES**

FIND OUT MORE: BIT.LY/HPVARTICLE

**Meyers, C., Milici, J., Robison, R. (2020) "The Ability of Two Chlorine Dioxide Chemistries to Inactivate Human Papillomavirus-contaminated Endocavitary Ultrasound Probes and Nasendoscopes" DOI: 10.1002/jmv.25666.

Duo Wipes are designed specifically for the application of Tristel Duo ULT.

Duo Wipes are made of 100% non-woven, low-linting polypropylene (18gsm) and pre-cut to prevent shedding when dispensed from the tub.

The use of Duo Wipes allows the effective release of the disinfectant onto the surface.



DUO WIPES

- 200 wipes per tub / 6 tubs per box

TRISTEL DUO ULT

- 125ml Tristel Base Solution (citric acid) + 125ml Tristel Activator Solution (sodium chlorite)
- When mixed upon pressing the foam pump, Tristel's proprietary chlorine dioxide chemistry is generated.
- 310 aliquots per bottle / 6 bottles per box

DUO DOK

- Designed to keep Tristel Duo ULT to hand
- 5 per pack

Tristel[™]
WE HAVE CHEMISTRY.

UK Manufacturer: Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

Hong Kong & Taiwan: Tristel Asia Limited, 21st Floor, 168 Electric Road, Fortress Hill, Hong Kong
T +852 2895 6968 - F +852 2869 4388 - E customerservicehk@tristel.com

*Based on virucidal efficacy validated in accordance with EN 14476.
For Tristel patent information please visit: <http://www.our-patents.info/tristel>

Copyright © Tristel Solutions
Mkt-Bro-1356-2
07/SEP/2022