

**HOW TO USE**



**1** Pour five litres of ambient water into a container.  
**Note:** Do not use hot water.



**2** Take one sachet, fold in half and squeeze one side to burst contents through the centre seal. Contents will start to turn yellow. Wait five minutes.



**3** Tear or cut the sachet.



**4** Pour the contents into the five litres of water.

**Note:** Take care when opening the sachet, do not splash concentrate. Keep away from face and eyes.




**5** For all blood and bodily fluid spillages, wipe up excess with disposable paper towels.



**6** Dispense into Tristel's one litre dosing bottles, then onto a cloth for surface disinfection or into a mop bucket.



**7** Apply **Tristel Fuse for Medical Surfaces** in an "S" shape with a cloth for surfaces. Leave to dry.

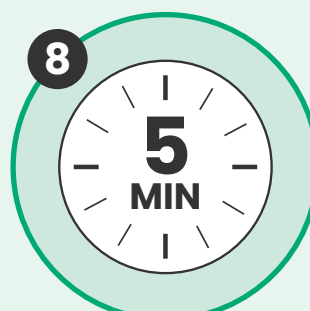

**8** Ensure a five minute contact time.

**NOTES**

- ✓ Make up solution in a well-ventilated area.
- ✓ Wear appropriate PPE. Refer to the Safety Data Sheet and to your hospital protocol.
- ✓ Store in original packaging in a cool, well-ventilated area out of direct sunlight.
- ✓ Once mixed with water, solution can be kept for up to 24 hours when stored in an air tight container.
- ✓ Dispose of empty packaging in accordance with local policy and national regulations.
- ✓ Avoid contact with skin and eyes. Contact with the disinfectant may cause mild irritation. Wash affected areas with plenty of soap and water.
- ✓ Rinse container with water after use.
- ✓ Do not use if packaging is damaged.
- ✓ For healthcare professional use only.