

HOW TO USE



1
 Pour 5 litres of ambient water into a container.
Note: Do not use hot water.



2
 Take one sachet.



3
 Fold sachet in half and squeeze one side to burst contents through centre seal. Contents will start to turn yellow. Wait 5 minutes.
Note: The dwell time is an important step in the product preparation and should be accurately followed.



4
 Tear or cut sachet. Take care when opening sachet, do not splash concentrate. Keep away from face and eyes.



5
 Pour contents into the 5 litres of ambient water. Product is ready to use.
Note: When pouring the contents of the sachet take care to keep the contents under the water line to avoid splashing.



6
 Apply to surface with mop for floors or a cloth for damp dusting other surfaces and leave to dry naturally for a 5 minute contact time.

NOTES

- ✓ Make up solution in a well ventilated area.
- ✓ Ensure appropriate PPE according to hospital protocol is worn prior to preparing **Tristel Fuse for Medical Surfaces**.
- ✓ Store in original packaging in a cool, well ventilated area out of direct sunlight.
- ✓ Once mixed with water, solution can be kept for up to 24 hours when stored in an air tight container.
- ✓ Dispose of empty packaging in accordance with local policy and national regulations.
- ✓ Avoid contact with skin and eyes. Contact with the disinfectant may cause mild irritation. Wash affected areas with plenty of soap and water.
- ✓ For healthcare professional use only.
- ✓ This wall chart should be used in conjunction with the instructions for use (IFU), product label, and Safety Data Sheet (SDS).