

Tristel Trio™

Wipes System

 Refer to the User Guide for full instructions.

Notes:

- Always follow hospital protocols and device manufacturers' instructions.
- Disinfect hands and wear appropriate PPE when handling disinfectants and medical devices.
- An initial pre-clean should be performed immediately after medical device use as per hospital protocol.
- Take care not to rub or bend the instrument and pay particular attention to edges, ridges, indentations and areas where different materials connect.
- Prepare for the decontamination process by taking one of each Wipe and the Activator Foam from the box.
- Do not use past the expiry date. For the expiry date and LOT number please see the surface of the carton.
- Do not use if the Wipe sachet is damaged, or the Wipe is discoloured, damaged or dry.
- Use more Wipes if required to ensure the instrument is visibly clean.
- Keep the Wipe sachets to complete the Tristel Traceability book at the end of the decontamination procedure.
- Discard the used Wipe and gloves in accordance with local regulations.
- For professional use only.
- This wall chart should be used in conjunction with the instructions for use (IFU), product label, Safety Data Sheet (SDS) and device manufacturer's instructions.

FOR HIGH-LEVEL
DISINFECTION OF SKIN
SURFACE PROBES

STEP 1 : CLEANING




Remove the Pre-Clean Wipe from its sachet and unfold it in the palm of your hand.

Wipe the holder, plug, cable and probe to remove all visible soil.

Discard the used Wipe and gloves in accordance with local regulations.

Disinfect your hands and put on new gloves.

STEP 2a + 2b : ACTIVATING

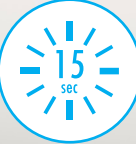

Remove the Sporidical Wipe from its sachet and unfold it in the palm of your hand.

Take the Activator Foam bottle and apply two doses of foam onto the Sporidical Wipe.

If the Activator Foam bottle is being used for the first time, depress the pump two to four times to prime it.

Fold the Sporidical Wipe in on itself and scrunch together **15 seconds** to generate chlorine dioxide.

Ensure that the Wipe is completely covered in Foam.

Use the activated Wipe immediately.

STEP 2 : HIGH-LEVEL DISINFECTING



Wipe the surface of the probe in one movement to cover it with foam, as well as the holder, cable and plug, ensuring all areas come into contact with the Wipe.

Discard the used Wipe in accordance with local regulations.

Observe a minimum contact time of **30 seconds**.

STEP 3 : RINSING



Remove the Rinse Wipe from its sachet and unfold it in the palm of your hand.

Wipe the holder, plug, cable and probe to remove excess foam.

Discard the used Wipe in accordance with local regulations.

STEP 4 : TRACEABILITY



Refer to the used Wipe sachets and Activator Foam bottle to complete the Tristel Quality Audit Trail Record Book.